


LYMPHATREX Expert

ARMSLEEVE


Anatomical Markers

- G** corresponds to the axilla (armpit)
- F** corresponds to the widest point of the biceps
- E** localized at the antecubital fossa
- D** corresponds to the widest point of the forearm
- C1** positioned at 6 cm from C
- C** corresponds to the narrowest point of the wrist



*The arm must be in a relaxed position.
Measure the heights on the inner side of the arm.*

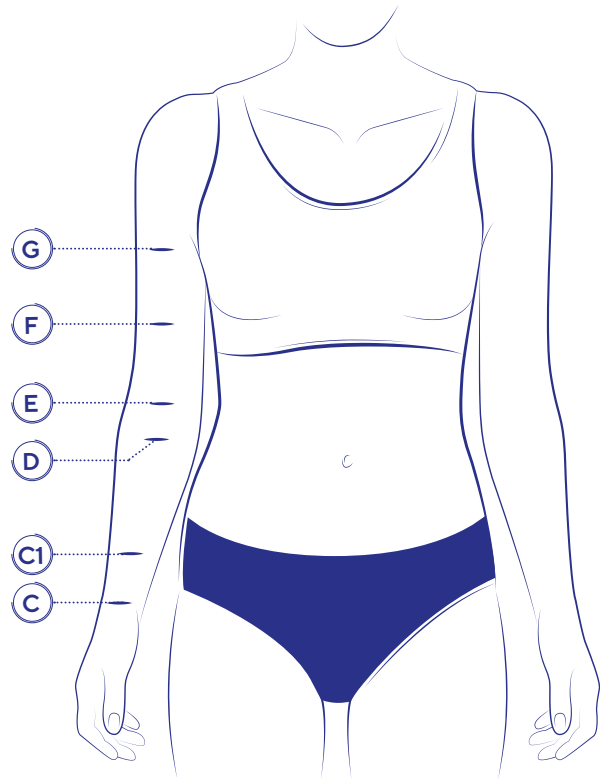


Measurement Instructions

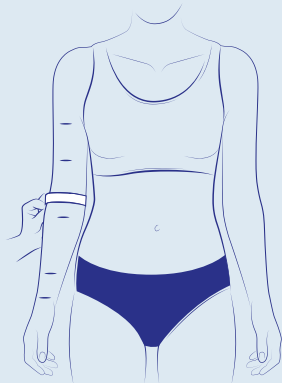


Measurement of circumferences (are illustrated here the fold of the elbow and the axilla)

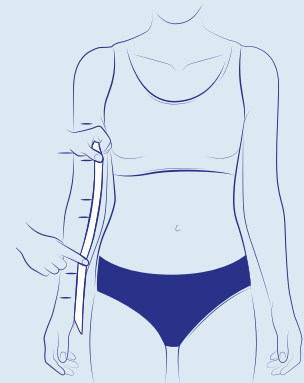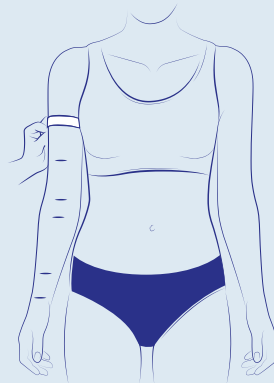

Important: the heights are taken on the inside of the arm

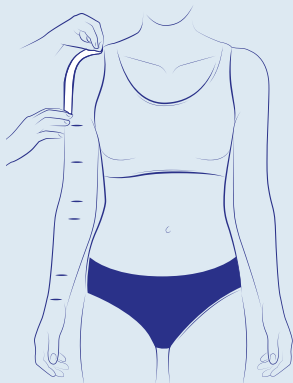

For a shoulder attachment, take the \angle GH measurement that goes from the axilla (projected on the outside of the arm) to the bra shoulder harness.

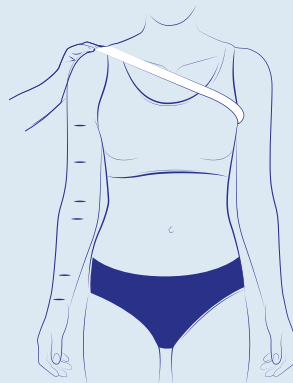

For a shoulder cover, take the chest circumference (H), from the acromion, passing through the opposite axilla.