


# LYMPHATREX Expert

## STOCKINGS




### Anatomical Markers

- (K) is located at the crotch (pantys/tights)
- (H) corresponds to the widest point of the buttocks (pantys/tights)
- (G) is located 5 cm under the gluteal fold
- (F) equidistant point between G and E
- (E) corresponds to the middle of the patella
- (D) is located on the head of the fibula (3 - 4 cm under the patella)
- (C) corresponds to the widest point of the calf
- (B1) corresponds to the start of the calf muscles
- (B) to be measured at the narrowest point of the ankle
- (Y) goes through the instep and the tuberosity of the calcaneus
- (A) goes through the heads of the metatarsal bones (at the widest point)



*All markers must be positioned on the external face of the leg.  
Do not forget to measure and indicate desired foot lengths (inner/outer).  
For a garment with open toe: please provide heel to toe tip measure.  
For a garment with closed toe: please provide heel to desired sitting step measure.*

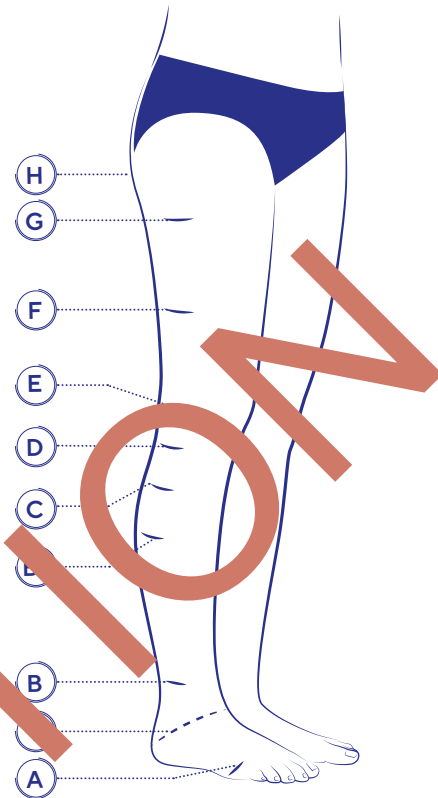


### Measurement Instructions

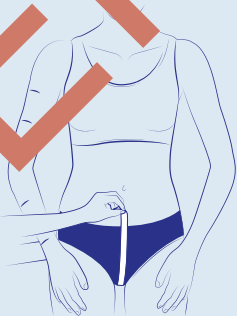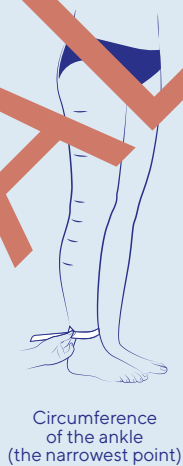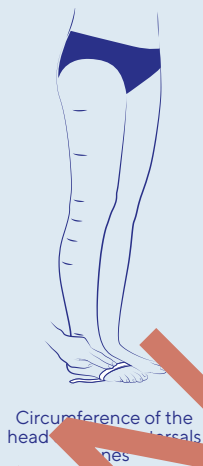

Panty/tights: the starting point for  $\ell_{GK}$ ,  $\ell_{GH}$  and  $\ell_{GT}$  is the pubic symphysis. These measures must follow the buttocks volume.

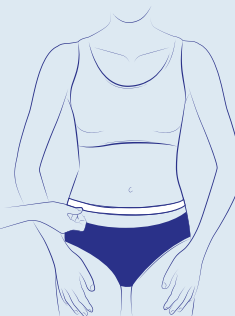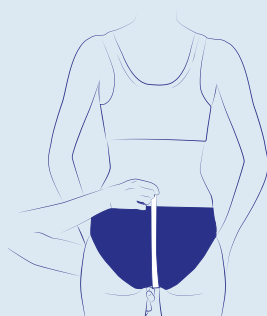

Circumference of the hips

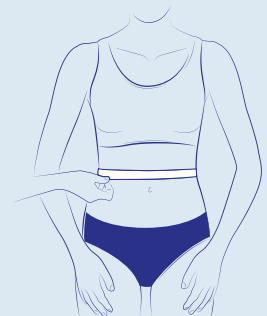

Circumference of the waist (or indicate the circumference at the desired height)