

LYMPHATREX Revolution

ARM SLEEVE

1- Anatomical markers




Select the anatomical markers required according to the garment model chosen.


Position the markers on the inner side of the arm.

- C** At the narrowest point of the wrist
- C1** At 6 cm from C
- D** At the widest point of the forearm
- E** At the antecubital fossa
- F** At the widest point of the biceps
- G** At the axilla (armpit)

*The arm must be in a relaxed position, with very slight flexion.
Important – measure the lengths on the inner side of the arm.*

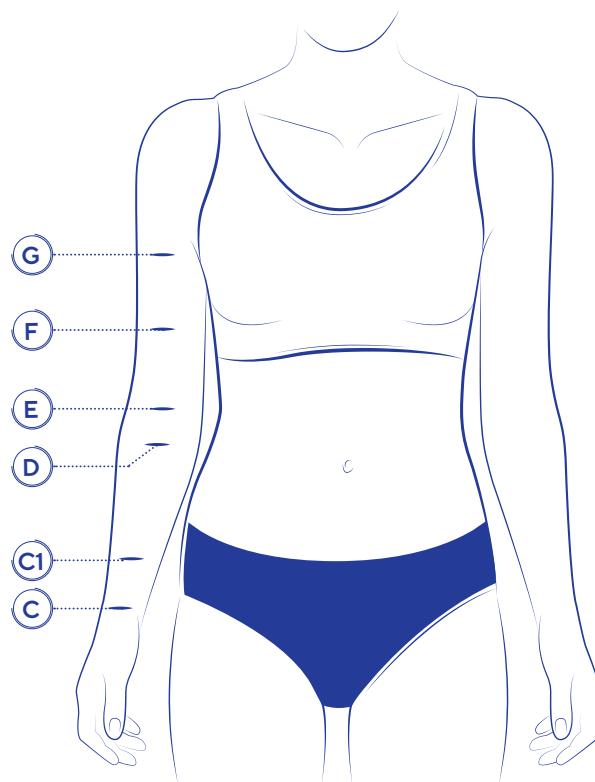

2- Circumferences measurement

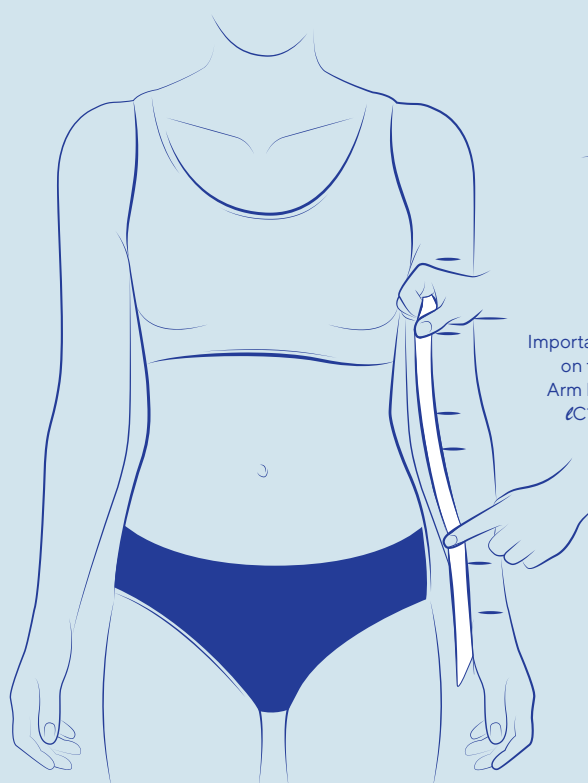

Important: the lengths are taken on the inside of the arm.
Arm lengths from the wrist :
ℓC1 (6 cm), ℓD, ℓE, ℓF, ℓG

