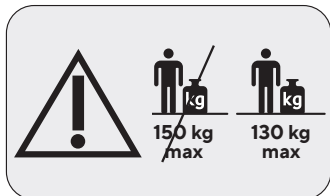
THUASNE®

Globe-Trotter W2016

Globe-Trotter+ W2017

ADDENDUM

fr

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en

Thank you for choosing the Thuasne Globe-Trotter (W2016) or Globe-Trotter+ (W2017) crutches.

IMPORTANT: Read this addendum and the instruction for use carefully before using this product and keep them for future reference.

WARNING:

Contrary to the information given in the instruction for use, the maximum user weight of the Globe-Trotter and Globe-Trotter+ crutches is 130 kg.

The healthcare professional will adjust the device and supervise its first use. Crutches are adjusted according to the user's height; **never lend them to another user without validation by a healthcare professional.**

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. **Do not use it for any other usage.**

MAINTENANCE & CARE

A lack of maintenance and care of your crutches **could result in serious injury.**

The **integrity** of crutches depends on the conditions of use, storage and compliance with the conditions of maintenance and care. **It is imperative to respect** the information provided in the instructions for use.

Always check the integrity of the device before each use. Do not use the device if it is damaged. Contact a specialist retailer or healthcare professional in the event of **damage.**

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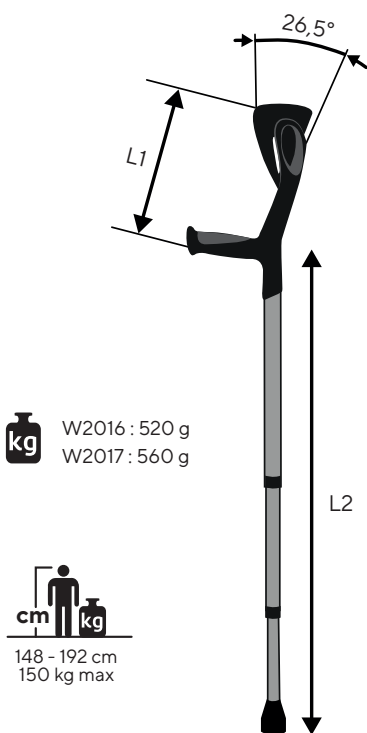
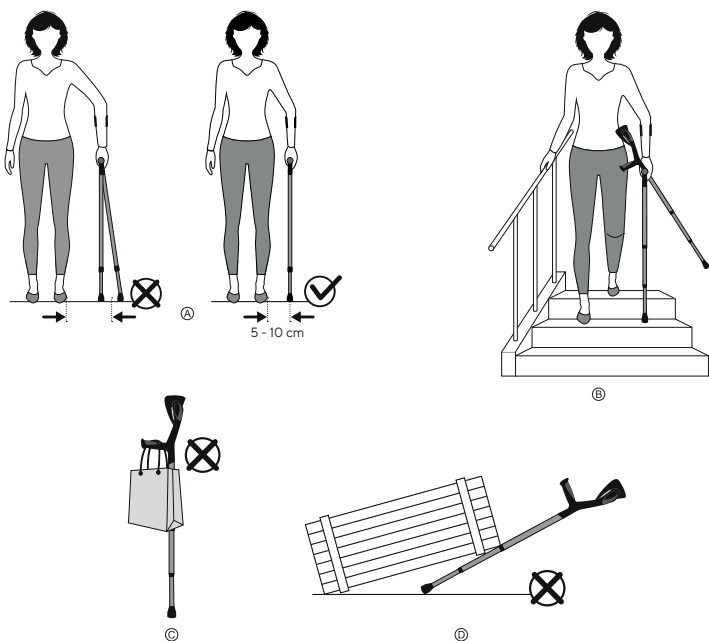
Description/Destination

Crutch.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It can be used inside and outside.

Main components

Tubes: aluminum.

Forearm support cuff: polypropylene.

Handle: polypropylene + thermoplastic elastomer (for the soft part)

Ferrule: PVC.

Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs. Degenerative conditions of the lower limbs.

Contraindications

Not suitable for patients weighing > 150 kg.

Do not use in the event of known allergy to any of the components.

Precautions

Important warnings

Verify the product's integrity before every use. Do not use the device if it is damaged.

It is recommended that a health care professional supervises the first use.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Before each use of this product, check that it is stable and rigid.

In case of any visible damage or significant wear of the ferrule, contact a specialized dealer or health care professional to have it replaced.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven or unstable surfaces (gravel, sand, leaves, etc.).

Do not leave exposed to direct sunlight for a long period of time.

Undesirable side-effects

- In case of incorrect use or overuse of the device: muscle fatigue, inflammation or injury. In case of incorrect height adjustment: skin reactions (redness, rashes, irritation, blisters, etc.) or wounds of varying severity, shoulder joint strain, pain in the hand, wrist, forearm or arm (biceps), hematomas and bruises, etc.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

Recommended user height: from 148 to 192 cm

The height adjustment of a crutch must be done with reference to the height of the patient. Incorrect height adjustment can impair performance and place too much strain on the user.

Crutch height adjustment:

After adjustment, when standing up straight with one hand on the handle, the arm should be bent at an angle of 30° (Fig. A).

Adjust using the clip: remove the clip and gently slide the lower tube into the upper tube.

Align the holes of both tubes until the handle is at a satisfactory height. Put the clip back in place.

Make sure that the crutch is locked in position by pressing it with increasing force on the ground.

Make sure that both crutches have been adjusted to the same height.

Use:

When walking, the crutch must be placed as vertically as possible on the ground (Fig. A). The crutch must be placed between 5 to 10 cm to the side, and approximately 15 cm in front of the foot (Fig A).

When in use, grasp the handle of the device completely and firmly.

In case of falling: Let the walking aid slip to the side so that you do not fall on it.

Important warnings

Always keep one hand on the handrail when going up or down stairs (Fig. B).

If possible, ask somebody for assistance.

When it is necessary to take stairs without a handrail, only do so with extreme caution.

Do not attach handbags, carrier bags or other objects to your device (Fig. C).

Only use the device as intended and not for any other use: pulling, lifting or moving objects (Fig. D).

Maintenance and care

Maintenance

If any part of the device becomes detached or loose, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from specialized dealers for the whole range of Thuasne products.

Care

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handle or the ferrule with greasy substances, this could create a slip hazard.

If necessary, use a mild contact disinfectant or 70° alcohol.

Storage

Store at room temperature.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 8657 092 001 (2024-04)