THUASNE

fr **Fauteuil roulant Classic DF 2**

W5310

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
W5310

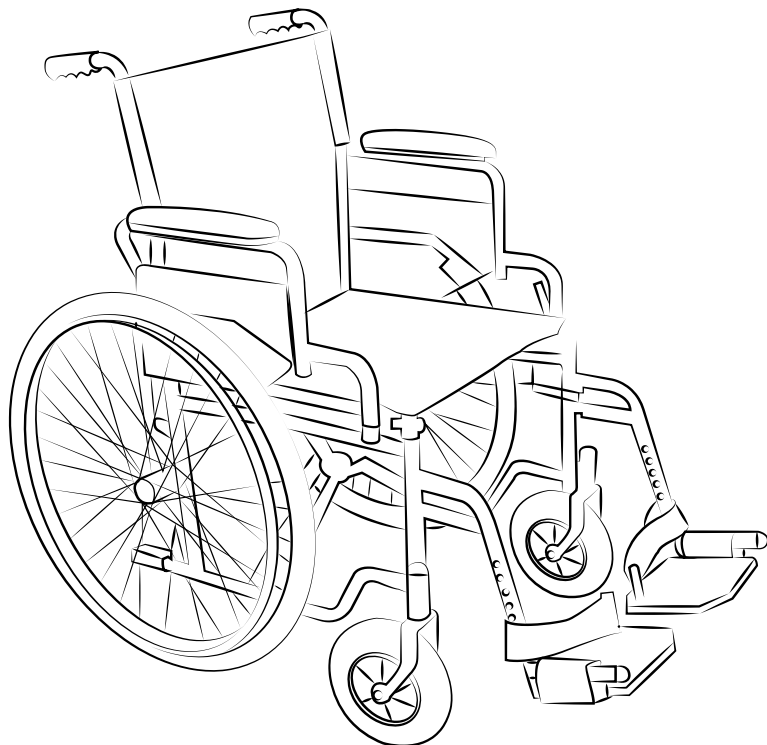
User and maintenance manual

This manual must be given to the user of the product.

Read this manual before using Wheelchair Classic DF 2

Keep this manual handy for future reference.

To obtain a copy of this manual, please contact your Thuasne dealer
(via the address www.thuasne.com/global-contact).

1. INTENDED USE	4	4.8. Footrest retraction/removal and fitting	10
1.1. Personnel interacting with the wheelchair	4	4.9. Adjusting the length of the footrest plates	11
1.1.1. Patient	4	4.10. Anti-tip (available from your dealer)	11
1.1.2. User	4	4.11. Retracting the leg-rests (accessory available from your dealer)	11
1.1.3. Medical professional	4	4.11.1. Removing and re-fitting	11
1.1.4. Technician	4	4.11.2. Adjusting leg-rest angle	12
1.1.5. Thuasne dealer	4	4.11.3. Folding the leg-rest pads	12
1.2. Description of the wheelchair	4	4.11.4. Adjusting the length of leg-rests	12
1.3. Operating Environment	4	4.12. Pelvic support belt (accessory available from your dealer)	12
1.4. Description of the intended patient group	5	4.12.1. Conditions of use	12
1.4.1. Target patients	5	4.12.2. Fixing the belt.....	12
1.4.2. Limitations of use	5	5. QUICK TROUBLE-SHOOTING	12
1.4.2.1. Cognitive ability	5	6. TECHNICAL CHARACTERISTICS	13
1.4.2.2. Physical ability	5	6.1. Standards and regulations	13
1.4.2.3. Selecting the wheelchair	5	7. LABELS	13
1.4.2.4. Required adjustments.....	5	7.1. Manufacturer's plate	13
1.5. Undesirable side-effects	5	7.2. Other label	14
2. BEFORE USING THE WHEELCHAIR	5	8. MAINTENANCE INSTRUCTIONS	14
2.1. Safety first	5	8.1. Tools required	14
2.2. Daily check	5	8.2. Maintenance rules	14
3. USING THE WHEELCHAIR	5	8.3. Cleaning	14
3.1. Nature of the ground	5	8.4. Disinfection	15
3.2. Wheelchair stability and limits	6	8.5. Inspecting the general condition of the wheelchair	15
3.2.1. Load and balance	6	8.6. Frequency of checks	16
3.2.2. Movements on the wheelchair.....	6	8.7. Long-term storage of the wheelchair	17
3.3. Transfer	6	8.8. Reusing the wheelchair	17
3.4. Reaching and picking up an object on the ground from your wheelchair	6	8.9. Expected life of the wheelchair	17
3.5. Obstacles	6	8.10. Recycling	17
3.5.1. Ramps and slopes - Cambers	6	8.11. Spare parts	17
3.5.2. Curbs	7	8.12. Accessories	17
3.5.3. Stairs.....	8		
4. BECOMING FAMILIAR WITH THE WHEELCHAIR AND ITS SETTINGS	8		
4.1. Unfolding and folding the wheelchair	8		
4.2. Transport and storage	9		
4.3. Armrests	9		
4.4. Parking brakes	9		
4.5. Solid tires	10		
4.6. Removing the wheels	10		
4.7. Adjusting the parking brakes	10		

After a study of the patient's needs and environment, a medical professional recommended this wheelchair and its accessories, also the procedures appropriate to the needs of the patient and users.

The wheelchair will be adjusted to your needs by a specialist. He/she will train you on the procedures to use it safely and efficiently.

If you have the slightest doubt concerning the safety or correct use of the wheelchair, you must contact your dealer and **do not use it** - in any circumstances-.

This manual gives instructions about using, assembling and maintaining the wheelchair and its associated limitations and dangers.

The use of the wheelchair presumes previous training, and a thorough knowledge and understanding of this user's manual. **Please read it carefully and fully before using the wheelchair.** Do not hesitate to refer to it whenever necessary.

For further information, please seek the advice of a healthcare professional.

If the wheelchair is used without an assistant, it should be used only by patients who are physically and mentally fit to control their actions safely.

The precautions for use presented in this manual are general warnings. If necessary, contact the Thuasne dealer or a healthcare professional and follow their usage and safety recommendations.

For more information about the product, safety advisories or product recalls, contact Thuasne representatives whose contact details can be found via the address www.thuasne.com/global-contact.

Maintenance of the wheelchair must be carried out by personnel with special training.

Thuasne reserves the right to make changes without prior notice. Therefore, it is possible that some parts of the text are not applicable, or illustrations are slightly amended.

 In this manual, the pictogram Warning indicates potentially dangerous situation(s) or hazardous practice(s). Carefully read the advice highlighted by this pictogram and, if in doubt, speak to your specialist about it.

Upon receipt of the wheelchair, please check for any damaged packaging and completeness of delivery. Complaints about visible damage must be made immediately.

A version printed in large typeface is available on request, via the address www.thuasne.com/global-contact.

1. INTENDED USE

1.1. Personnel interacting with the wheelchair

Only qualified personnel can judge the risks that could cause personal injury or property damage, and therefore operate

wheelchair functions safely. The various tasks described in this manual require various skills.

1.1.1. Patient

The patient is any person in need of care, or any handicapped or elderly person sitting in the wheelchair.

1.1.2. User

The user interacts with the functions of the wheelchair, and must have a clear understanding of the instructions for use and safety warnings described in this manual, as well as of possible hazards associated with the operation of those functions. For this purpose, he/she must carefully read and understand this manual. He/she may also benefit from the instructions and training dispensed by the medical personnel. Commonly-encountered users are: the assistant, the caregiver, the patient, as specified by the medical professional.

1.1.3. Medical professional

Qualified medical personnel trained for the application environment and with knowledge of the guidelines, rules and instructions relating to the use of wheelchairs.

After study of the patient's needs and the environment, the medical professional will specify the wheelchair, its settings and the possible accessories appropriate to the patient's and users' needs.

Your prescribing practitioner is a medical professional.

1.1.4. Technician

Professional personnel or organisations trained and experienced in the mechanical field, and with knowledge of rules and regulations concerning assembly, maintenance, and repair of the wheelchair. Your dealer is a mechanical professional.

1.1.5. Thuasne dealer

Your Thuasne dealer will adjust the wheelchair to the requirements of the prescription, check its mechanical performance, train you and your assistant(s) on its functions and safety precautions, check for any contraindication, and finally deliver the wheelchair.

1.2. Description of the wheelchair

The wheelchair is a manual folding wheelchair. It has been designed to offer comfort and autonomy to patients with temporarily or permanently impaired mobility.

Please refer to the diagram in chapter 4 of this manual "Becoming familiar with the wheelchair and its settings".

Non-inflammable materials and fabrics are used to manufacture the wheelchair.

1.3. Operating Environment

The wheelchair is designed for indoor or outdoor use, and can be handled alone or with the help of a person accompanying the patient.

This wheelchair is not designed to be used inside motor vehicles.

The wheelchair is not designed to be used with IV Pole, urine pouch, oxygen bottle, or cane-holder, as any one of these accessories could impact the wheelchair's stability.

Do not expose the wheelchair to temperatures higher than 39°C.

 **Do not expose the wheelchair to direct sunlight or to other heat sources. Such exposure can cause the temperature of the surfaces to rise, with a risk of the patient's skin being burnt. Before use, check the temperature of the surfaces in contact with the skin.**

Be careful when using cushions, blankets or weather protection that could be inflammable.

Do not use or store the wheelchair near naked flames or combustible products. Do not smoke when using the wheelchair.

If the wheelchair has been left out in bad weather, wipe it dry with a dry rag.

Follow the applicable traffic rules and follow the Highway Code.

The total weight on the wheelchair, including luggage, must not exceed 120 kg.

1.4. Description of the intended patient group

1.4.1. Target patients

The wheelchair is specially designed for adult patients age over 12 years, and a maximum weight of 120 kg.

It is intended for patients with temporarily or permanently impaired mobility who possess the necessary cognitive and motor capabilities to maintain a sitting position in a chair (trunk support). This impaired mobility may be due to a neurological problem, a genetic or cognitive deficiency, a disease of the lower limbs or trunk, or to age (limited endurance).

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

1.4.2. Limitations of use

1.4.2.1. Cognitive ability

The wheelchair must not be used for patients with psychiatric disorders.

The wheelchair must not be used for confused, agitated, disoriented patients, nor for patients with impaired judgment, unless a risk-assessment is carried out by a medical professional, and use of the wheelchair in such a condition is approved.

1.4.2.2. Physical ability

The consequences of certain physical deficiencies should be evaluated before prescribing the wheelchair: amputation of a lower or upper limb may lead to wheelchair imbalance.

1.4.2.3. Selecting the wheelchair

After study of your needs and the environment, your healthcare professional will write a prescription to specify the appropriate wheelchair, its settings and accessories, and your training programme.

He/she will consider the physical condition and training of your potential assistant(s), and/or carer(s).

1.4.2.4. Required adjustments

Before use, the wheelchair must be carefully adjusted to your needs. The mechanical professional makes the adjustments to the wheelchair according to the prescription. Your authorized dealer is responsible for a final check-up that the wheelchair with the chosen settings is fully safe for your use, and that you understand the manual's instructions.

Only a qualified person can make the adjustments. Incorrectly adjusting the wheelchair can lead to incidents resulting in serious injury.

1.5. Undesirable side-effects

- Risks associated with prolonged seated immobilization of the patient: pressure-related lesions, pressure ulcer formation, back pain (partly related to vibrations transmitted to the whole body).

- Risk related to repeated movements of the upper body as the patient uses the wheelchair: injuries such as carpal tunnel syndrome.

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

2. BEFORE USING THE WHEELCHAIR

2.1. Safety first

Safety requires constant attention by the person in the wheelchair and/or their helper.

It is extremely important to learn the safety rules, and always follow them during daily activities.

Consult your healthcare professional to determine the safety precautions and procedures that are best suited to your individual abilities, and receive training on appropriate methods to develop your individual technique.

The person who helps you should receive appropriate training and should be in good physical condition.

To prevent any injury, do not place fingers or any part of your body in moving parts.

As the maximum width of size 51 cm is greater than the recommended values, the user must be vigilant when using emergency exits.

2.2. Daily check

The wheelchair is designed to operate reliably and safely. However, like any mechanical device, it must be regularly checked. See paragraph 8.6. "Frequency of checks" in this manual.

Push-handle, tires, brakes must be checked each day. Poorly adjusted or non-operational elements could cause accidents.

3. USING THE WHEELCHAIR

3.1. Nature of the ground

Do not use your wheelchair on rough terrain, nor on soft ground (that is sandy, muddy, etc.). Do not use the wheelchair on slippery or icy surfaces.

Ensure that the way is clear of any obstacles.

If your wheelchair is fitted with an anti-tip device, ensure that the system is in place and is locked in the low position.

Adapt your speed to the environment (ground type, weather, etc.). Inappropriate operation can lead to loss of control of the wheelchair, collision, skidding or tipping and injury.

3.2. Wheelchair stability and limits

At the design stage, special attention was paid to the wheelchair's stability. However, it is possible to tip it over if used incorrectly.

3.2.1. Load and balance

Do not fasten or attach anything to the wheelchair. It could cause the wheelchair to tip over and might damage the wheelchair and/or cause injuries.

Do not use the wheelchair as a walking aid or rollator. Risk of loss of balance and injury.

3.2.2. Movements on the wheelchair

Do not lean backward to reach objects behind the wheelchair. Do not lean out sideways.

Do not, on your own, cause the front wheels (castors) of the wheelchair to lift off, and tilt the wheelchair backwards until it reaches its point of equilibrium. This practice is dangerous and could injure the user.

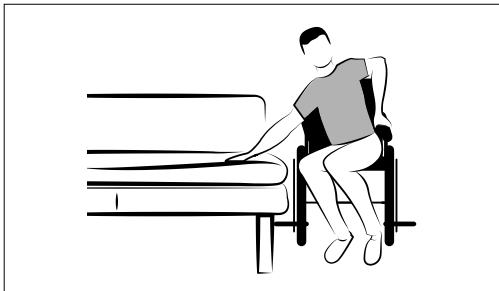
Do not place objects or containers on the tray table that could injure the user if they fall or tip over (e.g. hot liquids).

3.3. Transfer

Transferring into or out of a wheelchair is a very tricky operation. Take extra precautions to prevent any tipping. Whenever possible, have someone help you to perform this operation. Be extremely careful when transferring without outside help, or with assistance from a patient lift (hoist).

Lateral transfer

Before the transfer

- Bring the wheelchair as close as possible to the seat in which you wish to sit.
- Point the castors forward to improve stability.
- Make sure that the wheelchair is on a stable, flat surface, and lock the wheels.
- Ensure that the wheelchair is stabilised and will not shift during the transfer.

Fold up the footrest plates, and either remove them or swing them to one side.

During a transfer

Do not stand up or walk on footrests while transferring into or out of your wheelchair. That could cause the wheelchair to tip or to move unintentionally.

3.4. Reaching and picking up an object on the ground from your wheelchair

Manoeuvre the wheelchair as close as possible to the item to be reached. Swivel the two castors so that they point straight ahead.

Engage the two-wheel locks, your torso remaining upright on the seat of the wheelchair, and stretch out your arms towards the item to be reached.

If this operation does not enable you to grasp the object, change the position of the wheelchair or seek assistance.

Beware of being thrown off-balance by the weight of the item being picked up. If in doubt, ask for help or use a "reacher" or grabber.

Do not attempt to reach objects if you need to move forward on the seat. Do not attempt to retrieve items off the ground if you need to bend down between your knees.

3.5. Obstacles

Practice with a trained attendant or healthcare professional before attempting to pass obstacles unaided.

Assess your own capacities and know your limits in terms of strength and endurance.

Pavement kerbs, stairs and escalators are serious obstacles. Always try to avoid them by using a ramp or lift (hoist).

3.5.1. Ramps and slopes - Cambers

Before tackling a ramp, assess the potential risks: slippery or uneven surfaces, holes. If the entire ramp is not visible, ask a third party to inspect it.

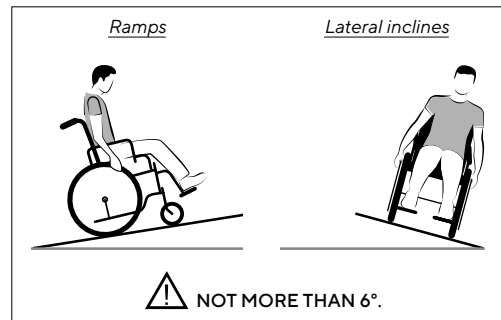
An anti-tip device is available from your dealer. If your wheelchair is fitted with that device, make sure that the anti-tip system is in place and is locked in the low position.


Do not turn the wheelchair by causing it to pivot while you are moving up or down a slope. It could tip over.

Do not venture onto inclines (ramps, slopes, lateral slopes) of more than six degrees (6°).


Going up a ramp

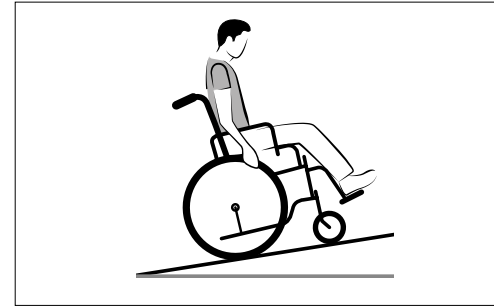- Lean your upper body forward slightly whilst going up the slope. To restart after a stop, avoid any sudden or sudden push on the handrails as this could tip the wheelchair.

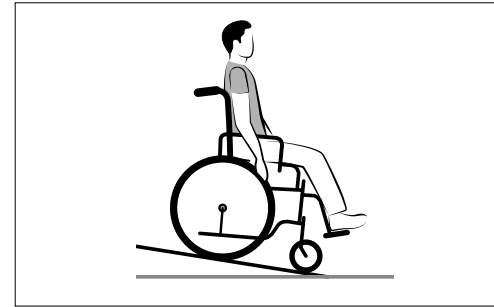Going down a ramp

When going down the ramp, keep footrests well clear of the ground. Lean slightly backward to improve stability. Go down slowly and with all due precaution by keeping hold of the handrims. Beware of heat given off by hands rubbing on the handrims.

Always go down the slope facing forward.

Never use parking brakes to slow your descent down. This could cause accidental locking of one or both wheels, and cause the wheelchair to tip over sideways or forward.

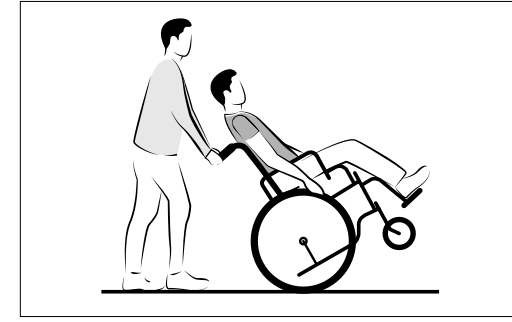
3.5.2. Curbs

If no ramp or lift (hoist) is available, ask another person to help you. The person who helps you should receive appropriate training.

Balancing and using the wheelchair in the equilibrium position requires good co-ordination between the attendant and user, and a certain degree of physical strength.


"Wheeling": balancing the chair on its rear wheels

Using both push-handles, the attendant inclines the wheelchair backward until it reaches its equilibrium position:

- the attendant can put some weight on the tilt pedal with the foot
- the person sitting in the wheelchair can help with a co-ordinated smooth move backward of the torso, while also pushing the handrim forward.

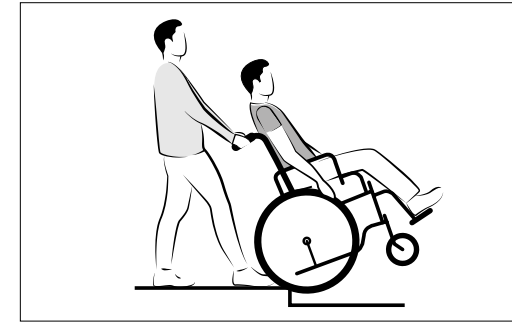
Curb: frontal descent

The attendant:

- tilts the wheelchair into its equilibrium position,
- then, while firmly holding the handles, pushes the wheelchair down the curb, facing forward,
- next, while firmly holding the handles, brings the front wheels gently back down.

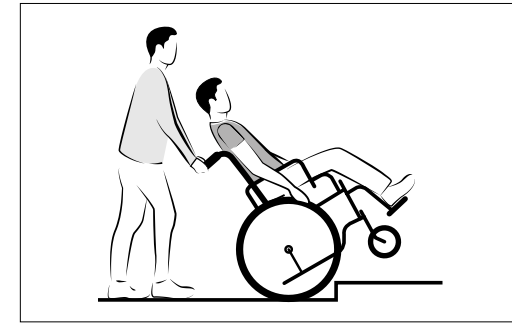Curb: frontal ascent


Take care not to hit the kerb with the footrests.

The attendant:

- tilts the wheelchair into its equilibrium position,

- pushes the wheelchair forward in order to bring the front wheels onto the pavement,
- then pushes and lifts it so that the rear wheels of the wheelchair will climb up the curb.

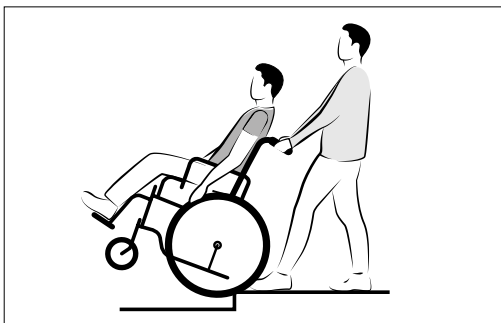Curb: reverse ascent

The attendant:

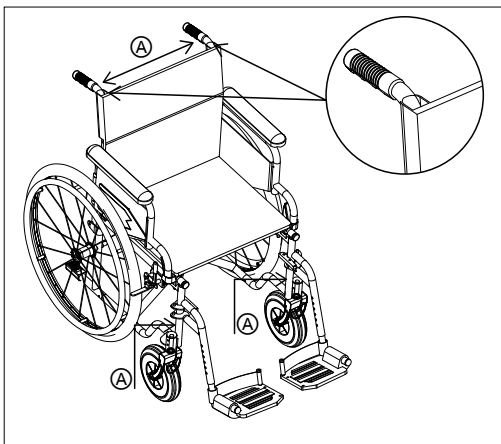
- turns the wheelchair round, so that the rear wheels are perpendicular to the curb,
- tilts the wheelchair into its equilibrium position, pulls the wheelchair up the kerb and, while holding the wheelchair in its equilibrium position, moves backwards to place the front wheels on top of the kerb,
- sets the wheelchair back down, while ensuring that the front wheels come firmly into contact with the ground.

3.5.3. Stairs


⚠ Never use an escalator to move a wheelchair between floors. Serious physical injury may occur.

Two experienced people are required to help the user in the wheelchair up or down stairs.

Carry points

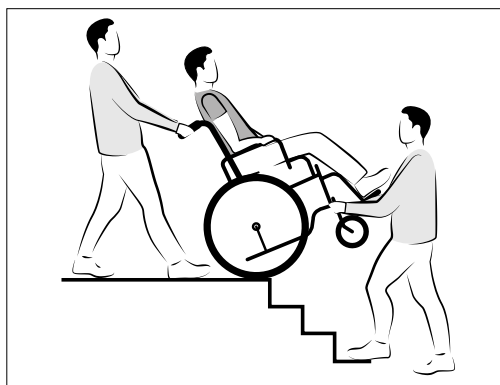
⚠ To carry or hold back the wheelchair, grip it by the points **A** shown on the diagram. Never carry the wheelchair by the armrests.

- Move one step at a time.
- The wheelchair should not be lowered until the last stair has been negotiated and the wheelchair has been rolled clear of the stairway.

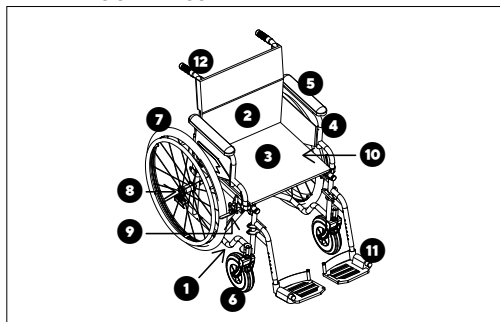4. BECOMING FAMILIAR WITH THE WHEELCHAIR AND ITS SETTINGS


- 1 Folding frame
- 2 Backrest upholstery
- 3 Seat upholstery
- 4 Armrest
- 5 Arm pad
- 6 Castor
- 7 Rear wheel
- 8 Quick-release pin for rear wheels removal
- 9 Parking brake
- 10 Carry strap (depending on the model)
- 11 Footrest
- 12 Push-handles

4.1. Unfolding and folding the wheelchair

The wheelchair can be folded to facilitate storage or transportation. Never lift the wheelchair by its mobile elements: leg rest, armrests, wheels, etc.

⚠ Do not place fingers between the seat and frame, pinch hazard. Use the palm of the hand to push it.

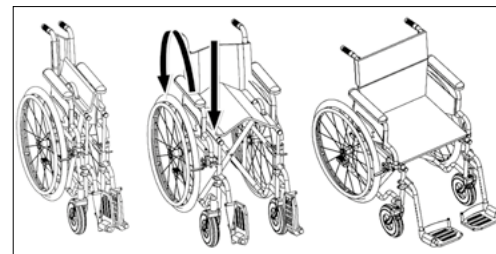
Unfolding the wheelchair

- Position the wheelchair with wheels on the floor/ground.
- Stand at one side of the wheelchair.
- Take hold of the push-handle on the side, and tilt slightly to lift the wheels on the opposite side.
- Press on the seat bar and unfold the cross-bar until the seat's upholstery is extended.
- Engage the right and left brakes and unfold the footplates.

⚠ Ensure that the seat is positioned properly on the supports.

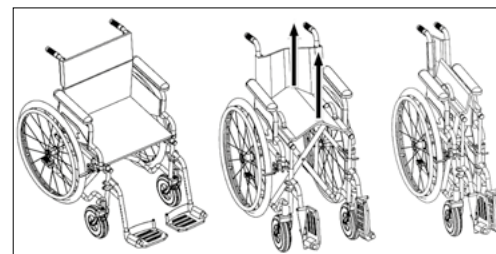Folding the wheelchair

- Remember to empty the rear pocket. Remove the seat cushion if it is being used.
- Fold the footrest or leg-rest plate back along the support tubes.
- Take hold of the two edges of the seat fabric at the centre, and lift.

4.2. Transport and storage

When folded, the wheelchair can be stored using less space, transported in the back of a car or inside the car boot.

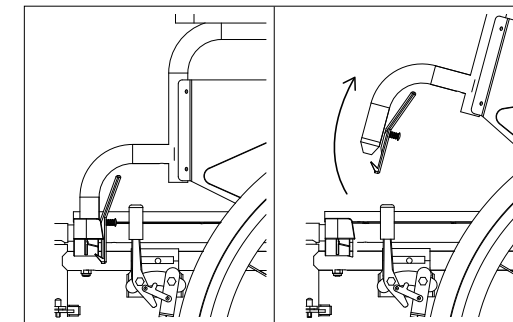
To handle it in its folded position, grip it by the carry points **A** (see paragraph 3.5.3 "Stairs").

It can be convenient for transport to remove the rear wheels if the wheelchair is fitted with quick-release wheels, and to remove the footrest(s) and/or leg-rest.

4.3. Armrests


The two armrests are retractable by swinging them away.

To swing the armrest up

- Push the locking lever forward.
- Then raise the armrest.

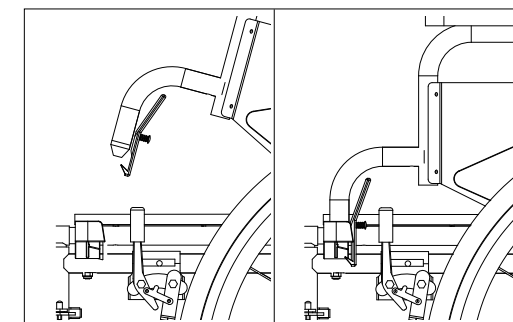To put the armrest back into position


- Swing the armrest down till the lock snaps into position.

⚠ Check the positive lock of the armrest by pulling it up.

⚠ Keep fingers away from moving parts when removing or fitting the armrest: pinch hazard.


4.4. Parking brakes

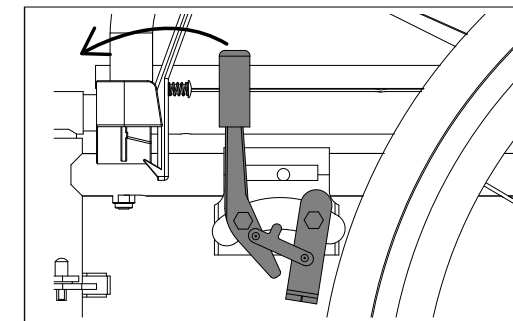To lock the rear wheels

Push the handles forward.

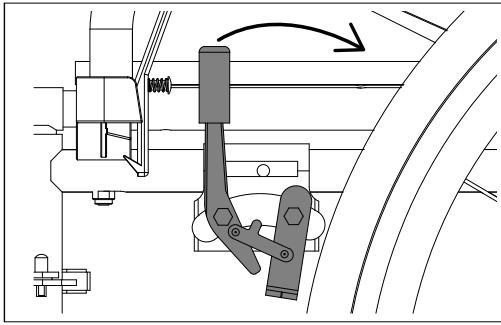To release the rear wheels

Pull the handles backward.

When applying brakes, keep hands and fingers on the brake handle. There is a risk of pinching or trapping fingers in the brakes' moving parts.

The handbrakes are not designed to slow your wheelchair down. The handbrakes are designed solely to hold the wheelchair in place when it is already at a complete standstill.

The brakes serve to hold the wheelchair stationary during extended stops. They should never be used to slow down the wheelchair as this could lead to loss of control or tipping.

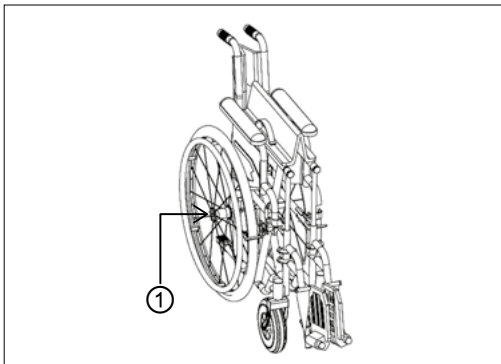
4.5. Solid tires

The solid tyres of the front and rear wheels will give long-lasting and trouble-free operation.

4.6. Removing the wheels

This operation can be performed if the rear wheel is fitted with a quick-release system. Depending on the operator's training and skill, various techniques can be used. This is one easy way of dismantling the rear wheel.

Removing the wheels

- Position the wheelchair in its folded position,
- Release the parking brakes
- Lock the armrest in down position.

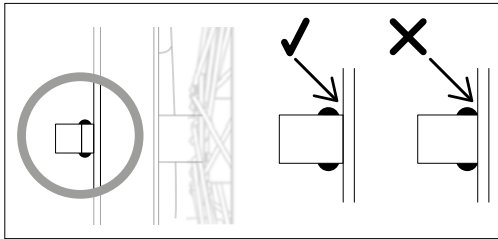
- With one hand, hold the wheelchair up. With the other hand's thumb, depress the central button of the quick-release device ①.
- Keep pressing, grasp the hub with the other fingers, and pull to remove the wheel.
- Position the wheel on floor or lean it against a wall.
- Hold the wheelchair in the upright position and repeat with the opposite wheel.

Putting the wheels back on

- The parking brakes should be in the released position.
- Depress the central button on the axle of each wheel ①, position it facing the hub and press in firmly.

Ensure that the wheel axles are fully engaged: the balls should extend from the bearing and lateral play must not exceed 2 mm.


4.7. Adjusting the parking brakes

This adjustment should be performed by a qualified professional. The adjustment should be performed on both brakes, rear wheels raised.

Loosen the locking nut of the brake mechanism so that it can slide over the side. Position the brake mechanism into the desired position. Tighten the nuts. Check that the brakes operate properly. If necessary, repeat the above steps until the brakes are properly positioned.


4.8. Footrest retraction/removal and fitting

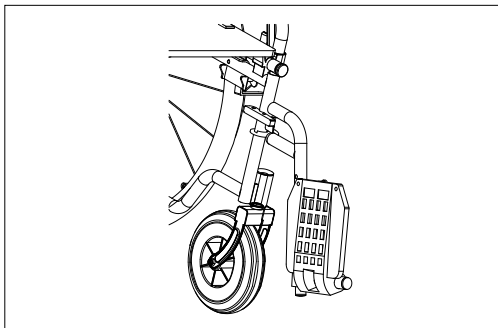Swinging away

- Apply the inside lever to unlock the mechanism and swing the footrest away outwards.

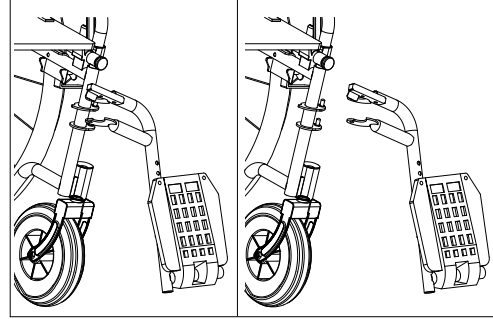Removal


- Extract the footrest from its 2 fastening pins or pivots.

Re-fitting:


Each footrest has a right side and left side. Do not swap them over. Align the holes in the brackets with the pins, and place the footrest on the frame supports. Swivel the support into the position at which it points forward from the wheelchair until the latch locks in position. Ensure that the latch is locked in position properly.

4.9. Adjusting the length of the footrest plates

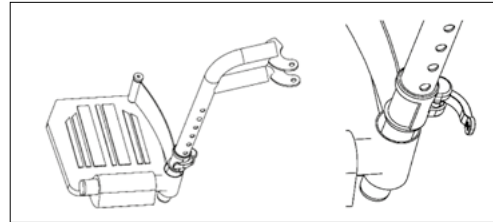
To be performed by a qualified professional.

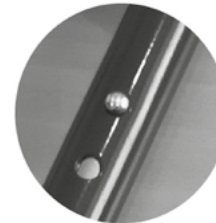- Unlock the lock device by placing the lever in its open position.

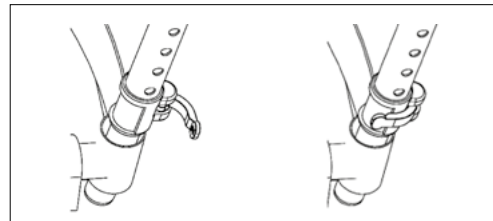- With both fingers, press the 2 silver locking clips, and adjust the plate to the desired height.

- Reinstal the safety lock by bringing the lever in its closed position.

Remember to maintain ground clearance of about 7 cm in order to prevent impacts with obstacles on the ground.

4.10. Anti-tip (available from your dealer)

Installation and adjustment must be performed by a qualified professional.

When in use, the anti-tip devices limit the backward tilt, reducing the risk of the wheelchair overturning. The devices are inserted inside the rear tubes of the frame and are locked in position with spring buttons.

Two positions are possible: the active position and the inactive position.

The anti-tipping system should be used on firm ground. Its action is diminished or canceled if the small wheels are not resting on firm ground.

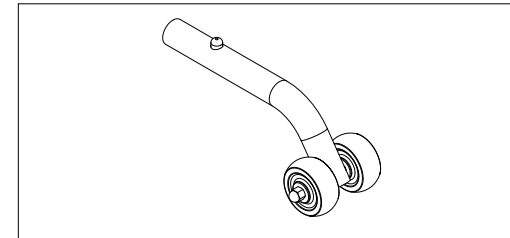
The anti-tipping system must be disabled before climbing or descending a step or curb as it could block when passing the obstacle.

The anti-tip function will be lost if the devices are not correctly set or are defective.

There is a risk of overturning if anti-tips are used on uneven surfaces, soft ground, potholes, etc.

Setting the active position of the anti-tip devices

- Remove the rubber foot cover from the frame.
- Press the spring button and insert the anti-tips.
- Position the devices with the small wheels close to the ground until the spring button snaps into position.
- Check that the spring button protrudes fully through the frame hole. Positive check that the device is locked in position.

Deactivating the anti-tip devices

- Depress the spring button and rotate the device with the wheels in the up position.
- Check that the spring button protrudes fully through the frame hole.
- Positive check that the device is locked in position.

There is a risk of overturning if the user is not aware of deactivation of the anti-tip device.

4.11. Retracting the leg-rests (accessory available from your dealer)

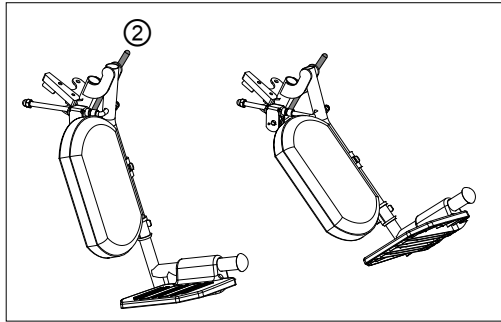
4.11.1. Removing and re-fitting

Since leg-rests are fastened identically to footrests, refer to paragraphs 4.8. and 4.9. concerning footrests.

Note: Each footrest has a right side and left side. Do not invert when re-fitting.


4.11.2. Adjusting leg-rest angle

Apply the holding lever ② and position the leg-rest at the desired angle. Release the lever.

4.11.3. Folding the leg-rest pads

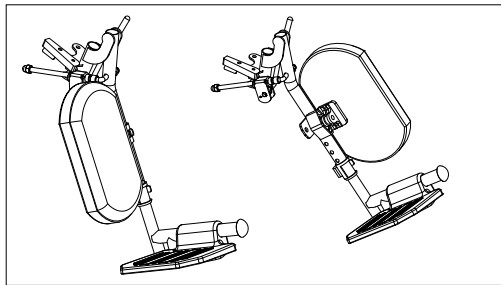Turn the pad forward from the wheelchair.

4.11.4. Adjusting the length of leg-rests

To be performed by a qualified professional.

Proceed in the same manner as for adjusting the length of the footrest supports.

Remember to maintain ground clearance of about 7 cm in order to prevent impacts with obstacles on the ground.

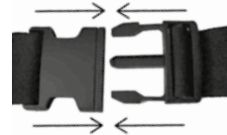
4.12. Pelvic support belt (accessory available from your dealer)

The wheelchair can be fitted with a pelvic support belt. The belt prevents the user from sliding down, or from falling out of the wheelchair.


4.12.1. Conditions of use

 Use of the belt must be specified by your medical specialist, and the user must be properly trained.

 In some situations, the belt must not be used. There are risks of strangling and/or serious injury should the patient slide under the belt.

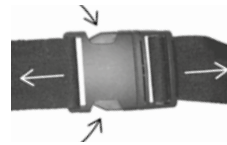Close the belt by inserting the retainers into the safety grip.


The quick release is obtained by pressing the side retainers.

Take extreme care when using a pelvic belt; check the belt correct adjustment at all times.

Adjusting the belt: to adjust the belt: sit the patient at the rear of the seat; with back resting straight on the back rest, the belt is adjusted properly when only one hand can be inserted between the belt and the patient's body.

4.12.2. Fixing the belt

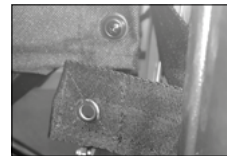Fixing the belt to the wheelchair frame must be done by your dealer. Swing both armrests backward.

Remove the bottom screws located at the left and right sides of the wheelchair's backrest.

Insert these screws through the grommets on the belt. Reinsert the screws into the screw holes and tighten them.

5. QUICK TROUBLE-SHOOTING

 If a wheelchair operating fault is detected, contact your dealer immediately.

To ensure correct wheelchair operation, the routine checks and maintenance must be performed.

If you detect any change in the behaviour of the chair, or a malfunction, put safety first and eliminate any obvious causes.

Some simple situations can be dealt with by the user, but most repair jobs must be carried out by your authorised dealer.

Detected trouble	Simple situation	Action
The wheelchair does not travel straight ahead.	Unexpected object, string, tangled round the front wheel axle.	Disentangle the object.
The wheelchair does not travel straight ahead. Rolling resistance is increased (i.e. difficult to push).	Defective or dirty bearings. Solid tyre is damaged. Wheels are not parallel.	Contact your dealer for repair.
Parking brake action is not effective.	Worn-out solid-tyre brake is not adjusted.	Contact your dealer for repair.
Wheelchair is difficult to fold or unfold.	Dirt on plungers.	Clean with wet cloth and dry thoroughly.

6. TECHNICAL CHARACTERISTICS

Model	cm	41	43	45	51
Length overall with footrest	mm	1075	1075	1075	1075
Width overall of the unfolded wheelchair	mm	622	642	662	722
Folded length with footrest	mm	1075	1075	1075	1075
Length without footrests	mm	775	775	775	775
Width of the folded wheelchair	mm	275	275	275	275
Height off the ground of the push-handles	mm	915	915	915	915
Weight of the wheelchair	kg	18.0	18.1	18.3	18.5
Weight of the frame with wheels and footrests removed	kg	10.9	11	11.2	11.5
Static stability downhill (front tilt)	deg.	20	20	20	20
Static stability uphill (back tilt)	deg.	14	14	14	14
Lateral static stability	deg.	18	18	18	18
Angle of the plane of the seat (backward)	deg.	3.8	3.8	3.8	3.8
Seat depth	mm	425	425	425	425
Seat width	mm	405	425	445	505
Sitting width between clothes guards (skirt guards)	mm	438	458	478	538
Distance ground to seat at the front/rear	mm	480 460	480 460	480 460	480 460

Model	cm	41	43	45	51
Angle of the backrest	deg.	14.5	14.5	14.5	14.5
Height of the backrest	mm	425	425	425	425
Footrest to seat distance* Height adjustable	mm	335 462	335 462	335 462	335 462
Angle between footrest and the plane of the seat	deg.	120	120	120	120
Armrest to seat distance	mm	260	260	260	260
Front location of armrest structure Desk position / Front position	mm	270 380	270 380	270 380	270 380
Handrim diameter	mm	20	20	20	20
Turning width (turning circle)	mm	1256	1256	1256	1256
Maximum safe gradient	deg.	6	6	6	6
Maximum slope for use of parking brakes	deg.	7	7	7	7

(*) 12 positions

Environmental parameters	Temperature: -20 °C to +40 °C	Humidity: 20% to 70%
--------------------------	-------------------------------	----------------------

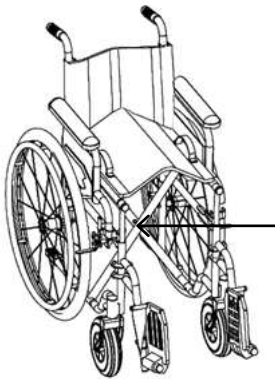
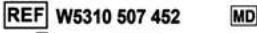
6.1. Standards and regulations

The wheelchair has been tested at CERAH and complies with NF EN 12182:2012 and 12183:2014. A 120 kg test dummy was used. This includes inflammability tests (EN 1021-1/2 and ISO 8191-2).

7. LABELS

7.1. Manufacturer's plate

Your wheelchair has its individual serial number, which appears on the manufacturer's plate, affixed to the frame of the wheelchair. This plate contains important information, useful for maintenance of the wheelchair.

Position and description	
	
  	
WHEELCHAIR CLASSIC DF2 45CM QR 24" BPU/B	
  	
Manufacturer and address	 THUASNE 120 rue Marius Aufan 92300 Levallois-Perret - France www.thuasne.com
Description	WHEELCHAIR CLASSIC DF2 45CM QR 24" BPU/B
Part number	REF W5310 507 452
Date of manufacture	
Batch number	LOT 201912
Serial number	SN XX XXXXX
Read manual before use	
Max. user weight	
CE mark	
Medical device	MD

7.2. Other label

This wheelchair is not designed for passenger transport inside motor vehicles.

This label is glued to the frame close to the identification label.

8. MAINTENANCE INSTRUCTIONS

8.1. Tools required

- Screwdriver: Phillips no. 2 (cross-head),
- Open-ended or hex wrenches: 8mm - 10mm - 13mm
- Hex wrench: 19mm.

8.2. Maintenance rules

Take care of your wheelchair: regular inspection, detecting and correcting problems, and replacing worn parts, will enable your wheelchair to continue to perform well for many years.

To prevent any injury, do not place fingers or any part of your body in moving parts.

In this section you will find a table summarizing the recommended inspections, their frequency, and the details of points to be checked.

If in doubt about how to make certain inspections, contact your dealer.

We recommend you have your wheelchair serviced by your dealer at least once a year.

If a wheelchair operating fault is detected, contact your dealer immediately.

Any repairs to your wheelchair should be done by your dealer, who has had special technical training.

Any unauthorised alteration, or use of spare parts other than those supplied by Thuasne, might affect the structure of the wheelchair, which could give rise to a dangerous situation causing serious injuries, and invalidate the warranty.

8.3. Cleaning

Do not use coarse abrasives, aggressive cleaning products or high-pressure cleaners. Never use organic solvents, acids or alkaline.

Your wheelchair is a mechanical vehicle, which is carefully designed and tested to give long-lasting performance. Cleaning is part of the regular maintenance of the wheelchair. In an institution, follow the recommendations of the hygiene manager. During cleaning, you might detect loose or damaged parts. Inform the dealer as soon as possible.

If the wheelchair gets dirty, wipe off the dirt as soon as possible with a damp cloth and dry it thoroughly. After using it in the rain, dry the wheelchair thoroughly with a cloth.

Seat fabric and backrest fabric

Clean with a wet rag.
For stubborn stains, use hot water and soap.
Do not soak the fabric, and do not use steam or aggressive solvents.

Plastic parts

Wipe the parts made of plastic, clean with a rag or a sponge. Use soap and water.

Surface coating: painted parts and anti-corrosion coating

The surface coating guarantees protection from corrosion. If the coating is damaged (chipped paint, scratches), have the damage repaired by your dealer.

For cleaning, use hot water and non-aggressive household detergents and a rag or sponge.

After any cleaning action

Rinse the surface with clean water.

Make sure you do not allow water or cleaning products to enter the tubes, and rinse surfaces thoroughly. Dry the surface by wiping down with a dry cloth.

Note: Sand or saltwater can damage functions, and steel parts can rust. If the wheelchair is exposed to a chlorinated or marine environment: clean it carefully, if necessary rinse the metal components with fresh water and wipe them dry.

8.4. Disinfection

The wheelchair can be disinfected by spraying or wiping with tested, approved disinfectants. Spray a soft cleaning and disinfecting product (bactericide and fungicide meeting the EN1040 / EN1276 / EN1650 standards), and follow the instructions given by the product supplier.

8.5. Inspecting the general condition of the wheelchair

Check for compliance of items to be inspected

Frame and components

Unfolding the wheelchair

The wheelchair should unfold easily and smoothly. The seat is positioned and rests uniformly on its supports. Plungers and seat guides move freely.

Structural integrity of the frame and components

Absence of buckling of tubes. No cracks or incipient fractures. No sharp edges or roughness.

Push-handles

These are bonded to the push-arms of the backrest and fastened securely. No cracking or signs of tearing.

Anti-corrosion coating

Good condition of the surface coating. Repair any defects in order to prevent risk of corrosion.

Nuts & bolts, screws

Use a screwdriver to ensure that screws are properly tightened. Use appropriate spanners to check nuts and bolts. Check that no burrs or sharp rough edges are present on screw heads or nuts, and replace if necessary.

Rolling and braking components

Wheels

The wheels should turn freely without jamming. Check for: buckling and play in the wheel axle.

If spokes are slack or loose, check or replace the wheel rim immediately.

Tire condition

Tires: no excessive wear, no cracking; adhesion of the solid tire to the rim.

Quick-release axles

Clean, locking balls operate firmly; axial play is limited.

Parking brake

No play: the parking brake should operate easily and smoothly. When placed "in position", the wheelchair is properly restrained.

Ball-bearings

On the rear wheels, front wheels, and on fork pivots: absence of play, structural integrity of the dust seals. Ball-bearings should be clean and should turn easily.

Fork pivots

No play, and angle relative to the vertical.

Check right-left symmetry. When rotated, the fork pivots spin freely. Dust cap not missing.

Checking rolling

When given a push, the wheelchair should move in a straight line without rubbing or resistance. No play in the wheel bearings.

Hand-rims

Fastened securely to the wheel; absence of play in the fastening.

No chipping in the surface treatment or any sharp edges or corners. In the event of any defect, replace the hand-rims immediately.

Seat

Upholstery of the backrest, upholstery of the seat

Tension of the fabric.

Retaining screws tightened properly. Absence of tearing in the fabric.

If the backrest fabric or seat fabric becomes slack due to intensive use, you should ask your dealer to change it: risk of tipping as the center of gravity is moved rearwards. Check that fastening screws are correctly and fully inserted and tightened.

No sharp roughness or any sign of looseness in the screws.

Armrests

Check that they operate easily and smoothly: unlock them and raise them. No excessive play, and no rubbing on the seat back.

Put them back in position, and check that they lock in position properly. Check the structural integrity of the clothes guards (skirt guards).

Make sure the screws to hold the arm pad do not penetrate the padding.

Footrest brackets and supports

Check they operate properly, and inspect the positions and the fastening of the plates.

Accessories (on certain models)

Anti-tip devices

Check that the anti-tip devices are fastened securely and positioned correctly. When the wheelchair is being used, these devices should be installed in the low position.

Leg-rest brackets and supports

Check they operate properly, and inspect the positions and the fastening of the plates. Check that the calf-pads are positioned properly.

8.6. Frequency of checks

Wheelchair	daily	weekly	monthly	Comments
Frame and components				
Unfolding the wheelchair		X		
Structural integrity of the frame and components		X		
Plastic parts		X		
Push-handles	X			
Anti-corrosion coating			X	
Nuts & bolts, screws		X		
Plungers clean: no corrosion		X		
Rolling and braking components				
Tire condition	X			
Quick-release axles		X		
Parking brakes	X			
Hand-rims		X		
Ball-bearings			X	
Fork pivots			X	
Checking rolling resistance			X	
When pushed, travels in straight line		X		

Wheelchair	daily	weekly	monthly	Comments
Seat				
Upholstery of the backrest, upholstery of the seat		X		
Armrests		X		
Footrest brackets and supports				
Checking locking		X		
Good condition of the plates		X		
Accessories (if fitted)				
Leg-rest brackets and supports		X		
Anti-tip devices		X		

8.7. Long-term storage of the wheelchair

Should you need to store the wheelchair for a prolonged period (for example, more than four months), follow these few simple steps:

- Carefully clean the chair, including upholstery and wheels, and dry all components.
- Keep brakes in unlocked position.
- Cover to protect against dust.
- Store in a dry location, protected from direct sun, and at room temperature.

After storage and before further use, carefully check the wheelchair following the "Inspecting the general state of the wheelchair" procedure (paragraph 8.5) described in this manual.

8.8. Reusing the wheelchair

Before re-using the wheelchair:

- Make sure the wheelchair is clean and disinfected,
- Carefully check the wheelchair following the "Inspecting the general state of the wheelchair" procedure described in the manual,
- Adjust to the new user.
- Provide the new patient with the user manual.

8.9. Expected life of the wheelchair

In normal conditions of use and with regular maintenance, the wheelchair's frame is designed to last at least 7 years.

Other parts that suffer normal wear-and-tear when the wheelchair is being used (upholstery of the seat or the backrest, arm-pads, solid tires, pneumatic tires, etc.), will need to be replaced as normal maintenance operations.

8.10. Recycling

Dispose of in accordance with local regulations.

8.11. Spare parts

Thusne spare parts are available from your dealer.

8.12. Accessories

Convenient accessories can be obtained from your dealer on order:

- Adjustable leg-rest
- Anti-tip systems.
- Pelvic support belt.

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©Thuasne - 2035801 (2022-07)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186