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#### en

### MEDICAL COMPRESSION

## SOCKS, STOCKINGS OR TIGHTS FOR WOMEN AND MEN

#### Description/Destination

VENOFLEX devices are intended for use on the lower limbs, by adults with the indications presented hereafter and whose measurements correspond to the size chart.

#### Composition

polyamide - élastane  
Components specific to certain product ranges (see label): viscose - linen - wool - cotton - silicone  
\* Silicone for the stocking format

#### Properties/Mode of action

VENOFLEX devices provide compression by applying pressure to the limb, thereby promoting venous return.

#### Indications

- 10-15 mmHg/Class 1: restless legs, heavy legs, evening oedema, reticular varicose veins, telangiectasia, prevention, during prolonged standing, long journeys (DVT prevention), pregnancy and post-partum (prevention and treatment of minor chronic venous insufficiency and DVT prevention), orthostatic hypotension.  
May be used in superposition/combination with other compression devices if higher pressure level is needed.
- 15-20 mmHg/Class 2: heavy legs, restless legs, evening oedema, telangiectasia or reticular veins (< 3 mm), following sclerotherapy for telangiectasia or reticular veins (< 3 mm), following sclerotherapy or surgery for varicose veins (> 3 mm), varicose veins (> 3 mm), prevention of venous thrombosis in pregnant women or post-partum, prevention of venous thrombosis during long journeys, treatment of lymphoedema (maintenance phase).
- 20-36 mmHg/Class 3: varicose veins (> 3 mm), following sclerotherapy or surgery for varicose veins (> 3 mm), chronic venous oedema, trophic disorders (pigmentation, eczema), chronic trophic disorders (lipodermatosclerosis or atrophie blanche), healed venous ulcer, chronic venous disease associated with pregnancy or the post-partum period, prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome, lower limb lymphoedema (maintenance phase).

#### Contraindications

Known allergies to any of the components.  
Advanced diabetic microangiopathy (if pressure > 30 mmHg).  
Heart failure.  
Weeping or severe dermatological conditions of the leg.

Peripheral artery disease with an ABI < 0.6.  
Plegmasia cereulea dolens (painful blue oedema with arterial compression).  
Septic thrombosis.

#### Precautions

Follow the advice of the healthcare professional who prescribed or supplied the product.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant restriction, pain, change in limb volume or unusual sensations, remove the product and seek the advice of a health professional.

Do not apply the product directly to an open wound.

Before using the stockings, make sure any infected or open skin lesions are covered by a suitable dressing.

Check that the anti-slip system can be applied to healthy, unbroken skin.

It is recommended that the product be washed before it is used for the first time.

Verify the product's integrity before every use.

For hygiene and performance reasons, do not re-use the product for another patient.

Venoflex Clinic: these products do not contain any metal wires liable to interfere with a magnetic field.

**Undesirable side-effect.**  
This device can cause skin reactions (redness, itching, blurs, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### Instructions for use/Application

Instructions for putting on: handle the product with care when putting on (pay attention to long nails and jewellery).

#### How to put on VENOFLEX socks

- 1 Place your hand inside the sock and pinch the heel
  - 2 Slip your foot into the sock until the heel is correctly positioned
  - 3 Pull up to below the knee
  - 4 Rub your leg to spread the sock evenly
- How to put on VENOFLEX FAST socks*
- 1 Turn back the top of the sock to the heel
  - 2 Slip your foot into the sock until the toes and heel are correctly positioned
  - 3 Unroll the sock from the ankle to below the knee

#### How to put on VENOFLEX SOFT&CARE socks

- 1 Pull over the heel
  - 2 Unroll the sock from the ankle to below the knee without pulling too much
  - 3 Rub your leg to spread the sock evenly
- How to put on VENOFLEX stockings or tight*
- 1 Place your hand inside the stocking and pinch the heel
  - 2 Slip your foot into the stocking or tight until the heel is correctly positioned
  - 3 Carefully pull up to the top of the thigh (do not pull too hard on the anti-slip band), or to the waist for tights
  - 4 Rub your leg to spread the stocking or tight evenly

#### Care

Machine-washable without fabric softener. Dry in the shade under direct sunlight or direct heat.

#### Storage

Store at room temperature, preferably in the original packaging.

#### Disposal

Dispose of in accordance with local regulations. Keep this instruction leaflet.

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©Thuasne - 2028201 (2021-06)