


THUASNE®

en	Sport reinforced patellar knee brace	4
fr	Genouillère rotulienne renforcée sport	5
de	Verstärkte Sport-Kniebandage für die Kniescheibe	7
nl	Verstevigde kniebrace voor de knieschijf sport	8
it	Ginocchiera rotulea rinforzata per uso sportivo	10
es	Rodillera rotuliana reforzada para uso deportivo	11
pt	Joelheira rotuliana reforçada para uso desportivo	13
da	Forstærket knæskalsbind sport	14
fi	Vahvistettu polvilumpiotuki urheilukäyttöön	16
sv	Förstärkt knäskålstöd sport	17
el	Αθλητική ενισχυμένη επιγονατίδα	19
cs	Zpevněná česková ortéza sport	20
pl	Orteza kolanowa do rzepki, wzmocniona sportowa	22
lv	Sporta ceļgala saišu ortoze ar pastiprinājumu	23
lt	Sutvirtintas kelio girnelės įtvirtas sportui	25
et	Spordi põlveketra toetav tugevdatud põlvekaitse	26
sl	Ojačena kolenska opornica za pogačico šport	28
sk	Športová zosilnená kolenná ortéza	29
hu	Megerősített patella térdrögzítő sportoláshoz	30
bg	Подсилена пателарна наколенка за спорт	32
ro	Genunchieră de susținere rotuliană ranforsată sport	34
hr	Pojačani sport steznik za patelu	35

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

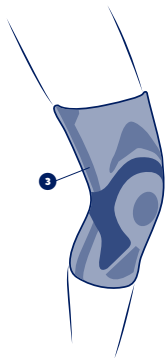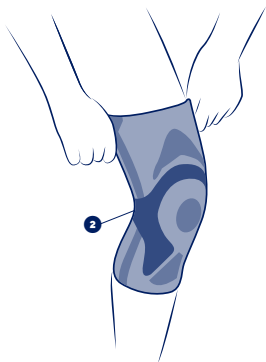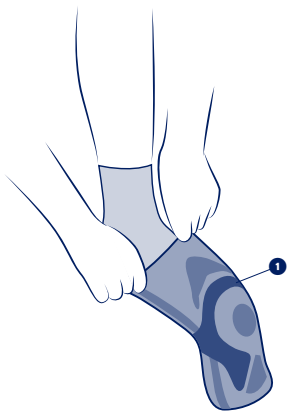cm
S
32 - 34
M
35 - 37
L
38 - 41
XL
42 - 45
XXL
46 - 49

en	Knee circumference in slight flexion
fr	Circonférence de genou en légère flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooid knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenktos kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
hr	Obuhvaća koljeno u blagoj fleksiji

SPORT REINFORCED PATELLAR KNEE BRACE

Support level 2

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of:

- a patellar insert ❶,
- an anatomically-shaped knitting integrating a specific knit at the back of the knee ❷,
- flexible lateral stays ❸.

This device can be used when resuming sports activities.

Composition

Textile components: polyamide - polyester - polyurethane - elastane - silicone - ethylene vinyl acetate.

Rigid components: steel - polyethylene.

Properties/Mode of action

An anatomically-shaped knitting for even compression and proprioception.

The knee support is maintained on the leg thanks to flexible side stays and anti-slip system.

Indications

Anterior knee or patella pain.

Mild patellar instability.

Patella tendinopathy (including jumper's knee syndrome).

Knee pain and/or swelling management.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the product on (the logo indicates the top and front of the product).

Make sure that your knee cap is positioned in the centre of the patella ring.

Care/Maintenance

Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2049001 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186