THUASNE

ImmoClassic

- fr** Écharpe d'immobilisation d'épaule
- en** Shoulder immobilisation sling
- de** Tuch zur Ruhigstellung der Schulter
- nl** Sling voor de immobilisatie van de schouder
- it** Fascia di immobilizzazione della spalla
- es** Cabestrillo para inmovilización del hombro
- pt** Suportede imobilização do ombro
- da** Halstørklæde til skulderimmobilisering
- fi** Olkapään immobilisaatiokantoside
- sv** Halsduk för att immobilisera axeln
- el** Φάκελος ακινητοποίησης ώμου
- cs** Imobilizační ramenní závěs
- pl** Szal do unieruchomienia barku
- lv** Pleca imobilizācijas soma
- lt** Peties imobilizavimo įtvaras
- et** Õlaliigest fikseeriv side
- sl** Zanka za imobilizacijo rame
- sk** Ramenný imobilizačný záves
- hu** Vállrögzítő heveder
- bg** Бандаж за обездвижане на рамото
- ro** Orteză „tip eşarfă” de imobilizare a umărului
- ru** Повязка для обездвиживания плеча
- zh** 肩部固定围巾

	1	50 - 71 cm
	2	72 - 94 cm
	3	95 - 117 cm
	4	118 - 140 cm

THUASNE
120, rue Marius Aujan
92300 Levallois-Perret - France

© Thuasne - 6165601 (2021-04) © Studio Caterin


www.thuasne.com

01/23 07/201

	1	50 - 71 cm
	2	72 - 94 cm
	3	95 - 117 cm
	4	118 - 140 cm

fr	Tour de taille	Immobilisation
en	Waist measurement	Immobilisation
de	Tailenumfang	Ruhigstellung
nl	Tailleomvang	Immobilisatie
it	Circonferenza vita	Immobilizzazione
es	Contorno de cintura	Inmovilización
pt	Perímetro de cintura	Imobilização
da	Taljemål	Støtte
fi	Vyötäröympärys	Immobilisoi
sv	Midjemått	Immobilisering
el	Περίμετρος μέσης	Ακιντονόισον
cs	Obvod pasu	Znehybnění
pl	Obwód w talii	Unieruchomienie
lv	Vidukļa apkārtmērs	Imobilizācija
lt	Juosmens apimtis	Imobilizavimas
et	Vööümbarmõõt	Liikumatuks muutmine
sl	Obseg pasu	Imobilizacija
sk	Obvod pása	Imobilizácia
hu	Derékbőség	Rögzítés
bg	Обиколка на талията	Обездвижане
ro	Circumferința taliei	Imobilizare
ru	Обхват талии	Иммобилизация
zh	腰围	固定

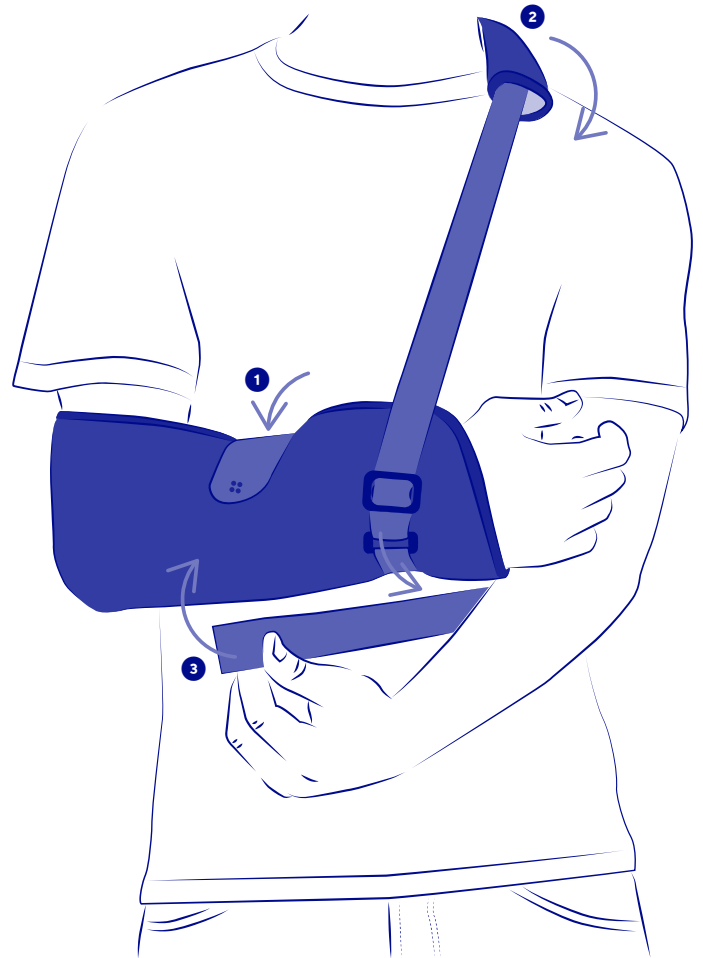MD

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SHOULDER IMMOBILISATION SLING

Description/Destination

This device is a shoulder sling allowing the immobilisation of the shoulder and/or the elbow in neutral position.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

polyurethane - polyester - polypropylene - polyamide - cotton

Properties/Mode of action

Foam sling to support forearm and elbow.

Adjustable strap system allowing immobilisation in neutral position.

Adjustable foam pad for comfort and protection of the neck.

Bilateral product.

Indications

Shoulder and/or elbow injuries or inflammatory conditions.

Conservative treatment for non-surgical proximal humeral fractures.

Shoulder dislocation.

Non surgical fractures of the clavicaula and/or scapula.

Post-operative immobilization (such as after rotator cuff surgery or dislocation reduction...).

Shoulder subluxation or dislocation prevention (joint instability). Support for paralytic upper limb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of complex or surgical shoulder fractures or rib cage fractures.

Do not use in certain postoperative conditions (multiple fractures, pin insertion, open fractures, etc.).

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Do not wear the product when driving a vehicle.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

❶ Place the forearm in the sling. Tighten the sling around the forearm using the self-fastening tab.

❷ Place the upper strap around the opposite shoulder. Place the strap in front of the sling and attach it on the external side of the sling. Adjust the strap length.

❸ Place the lower strap around the waist. Attach it under the arm on the external side of the sling.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros