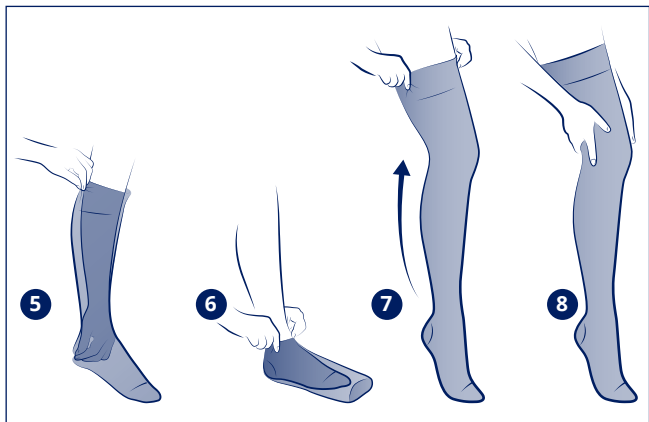
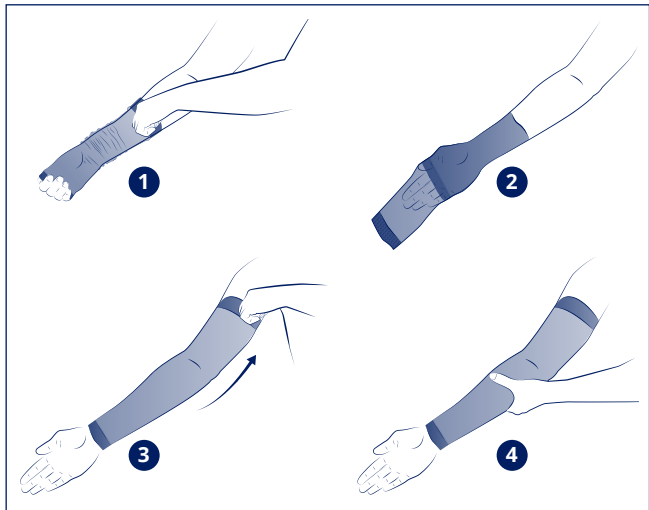
THUASNE

LYMPHATREX Expert

fr	Dispositif de compression sur mesure patient.....	4
en	Made-to-measure patient compression device.....	6
de	Für Patienten maßgefertigtes Kompressionsprodukt.....	8
nl	Compressiehulpmiddel op maat van de patiënt.....	10
it	Dispositivo di compressione su misura paziente.....	13
es	Dispositivo de compresión a medida del paciente.....	15
pt	Dispositivo de compressão à medida do paciente.....	17
da	Kompressionsprodukt efter patientens mål.....	20
fi	Kompressiotuote potilaan mittojen mukaan.....	22
sv	Individanpassad kompressionsprodukt.....	24
el	Διάταξη συμπίεσης στα μέτρα του ασθενούς.....	26
cs	Kompresivní pomůcka na míru pacienta.....	29
pl	Wyrób uciskowy wykonywany dla pacjenta na miarę.....	31
lv	Pacientam pielāgots kompresijas izstrādājums.....	33
lt	Pagal paciento užsakymą pagaminta kompresinė priemonė.....	36
et	Patsiendile eritellimusel valmistatud kohandatud kompressioontoode.....	38
sl	Kompresijski pripomoček po meri bolnika.....	40
sk	Kompresívna pomôcka vyrobená na mieru pacienta.....	42
hu	A páciens méretére készült kompressziós eszköz.....	44
bg	Персонализирано компресионно изделие за пациента.....	47
ro	Dispozitiv de compresie adaptat la măsura pacientului.....	49
ru	Компрессионные изделия по меркам пациента.....	51
hr	Kompresijski proizvod po mjeri bolesnika.....	54
zh	为患者定制的压力矫形器.....	56
ja	オーダーメイドの圧迫医療器具.....	58
ko	환자 맞춤형 압박스타킹.....	60
ar	لباس ضاغط على مفاص المريض.....	61

fr	Dispositif médical	Un seul patient - à usage multiple
en	Medical device	Single patient - multiple use
de	Medizinprodukt	Einzelner Patient - mehrfach anwendbar
nl	Medisch hulpmiddel	Eén patiënt - meervoudig gebruik
it	Dispositivo medico	Singolo paziente - uso multiplo
es	Producto sanitario	Un solo paciente - uso múltiple
pt	Dispositivo médico	Paciente único - várias utilizações
da	Medicinsk udstyr	Enkelt patient - flergangsbrug
fi	Lääkinnällinen laite	Potilaskohtainen - voidaan käyttää useita kertoja
sv	Medicinteknisk produkt	En patient - flera användningar
el	ιατροτεχνολογικό προϊόν	Πολλαπλή χρήση - σε έναν μόνο ασθενή
cs	Zdravotnický prostředek	Jeden pacient - vícenásobné použití
pl	Wyrób medyczny	Wielokrotnie użycie u jednego pacjenta
lv	Medicīniska ierīce	Viens patients - vairākkārtēja lietošana
lt	Medicinos priemonė	Vienas pacientas - daugkartinis naudojimas
et	Meditsiiniseade	Ühel patsiendil korduvalt kasutatav
sl	Medicinski pripomoček	En bolnik - večkratna uporaba
sk	Zdravotnícka pomôcka	Jeden pacient - viacnásobné použitie
hu	Orvostechnikai eszköz	Egyetlen beteg esetében többször újrahasználató
bg	Медицинско изделие	Един пациент - многократна употреба
ro	Dispozitiv medical	Un singur pacient - utilizare multiplă
ru	Медицинское изделие	Многократное использование для одного пациента
hr	Medicinski proizvod	Jedan pacijent - višestruka uporaba
zh	醫療器械	一位患者 - 多次使用
ja	医療機器	一患者用 - 複数回使用
ko	의료 기기	한 환자에 한 해 - 여러번 사용
ar	جهاز طبي	لمريض واحد - متكرر الاستعمال

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en MADE-TO-MEASURE PATIENT COMPRESSION DEVICE

Description

LYMPHATREX Expert is a range of compression devices made to the measures of the patient's limb by flat knit.

This range is available in several models:

Upper limb:

- Armsleeve
- Armsleeve with mitten
- Armsleeve with glove
- Mitten
- Glove

Lower limb:

- Sock
- Stocking
- Half-tight
- Tights
- Panty
- Toe cap

Various finish options are available, depending on the recommendations of your healthcare professional.

Composition

Lower limb: polyamide, elastane, polyester, cotton

Upper limb: polyamide, elastane, polyester, cotton

Anti-slip system: silicone, elastane, polyamide, polyester

Compression pad option: ethylene-vinyl acetate

Mechanism of action

LYMPHATREX Expert devices provide compression by applying pressure to the limb; they thus promote lymph circulation, help prevent oedema formation and maintain its volume.

They are available in multiple AFNOR compression classes:

- Upper limb
 - Class 2 (15-20 mmHg)
 - Class 3 (20-36 mmHg)
- Lower limb
 - Class 3 (20-36 mmHg)
 - Class 4 (> 36 mmHg)

Destination: The device is intended to be used directly on the skin of the lower or upper limbs, in adult or child patients suffering from at least one of the indications listed below.

Indications

LYMPHATREX Expert compression devices are indicated in the following cases:

- Treatment of acute or chronic edema, for example primary or secondary lymphedema of the upper or lower limbs
- Venous diseases of the lower limbs (the recommended compression classes are indicated in brackets):
 - Varicose veins (≥ 3 mm) (20 - 36 mmHg)
 - Following sclerotherapy or surgery for varicose veins (≥ 3 mm) (20 - 36 mmHg)
 - Chronic venous edema (20 - 36 mmHg)
 - Trophic disorders: pigmentation or eczema, chronic lipodermatosclerosis or chronic atrophie blanche (20 - 36 mmHg)
 - Healed venous ulcer (20 - 36 mmHg and > 36 mmHg)
 - Active venous ulcer (> 36 mmHg)
 - Chronic venous disease associated with pregnancy or post-partum (20 - 36 mmHg)
 - Prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome (20 - 36 mmHg).

Fitting instructions

- Check the product is not damaged before use.

• LYPHATREX Expert devices should be worn next to the skin.

• Seek advice from a healthcare professional.

Putting on the armsleeve, armsleeve with mitten, armsleeve with glove:

• Place your hand inside the garment ① and turn it back on itself to the hand opening. ②

• Place your hand inside the garment, then carefully slide it over the limb to under the armpit (armsleeve, armsleeve with mitten and armsleeve with glove), or to the wrist (mitten and glove). ③

• Avoid pulling the garment any more than necessary.

• Adjust the garment to ensure there are no wrinkles along the length of the limb.

• Once the garment is properly in place, massage the limb to ensure uniform compression. ④

Putting on the mitten, glove and toe cap:

• Place your hand/foot inside the garment, then carefully slide it to the wrist (mitten and glove) or ankle (toe cover).

• Avoid pulling the garment any more than necessary.

• Adjust the garment to ensure there are no wrinkles on the hand/foot.

• Once the garment is properly in place, massage the limb to ensure uniform compression.

Putting on the socks, stockings, tights, half-tights and panty:

• Place your hand inside the garment and turn it back on itself for the panty and to the heel for the other models. ⑤

• Slip your foot into the garment until the heel is correctly positioned. ⑥

• Carefully slide the garment over the limb to below the knee (sock), below the gluteal fold (stocking), or to the waist (tights, half-tights and panty). ⑦

• Avoid pulling the garment any more than necessary.

• Adjust the garment to ensure there are no wrinkles along the length of the limb.

• Stocking or sock with anti-slip system: when putting on the garment, be careful not to grip the anti-slip part to avoid any damage.

• Once the garment is properly in place, massage the limb to ensure uniform compression. ⑧

• Phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of a limb concerned

• Septic thrombosis of a limb concerned

Do not use LYPHATREX Expert on the lower limbs in the following cases:

• Peripheral artery disease (PAD) with ankle-brachial pressure index (ABPI) < 0.6

• Decompensated heart failure

• Advanced diabetic microangiopathy (for compression > 30 mmHg)

• The risk/benefit ratio must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9

- Advanced peripheral neuropathy

Do not use LYPHATREX Expert on the upper limbs in the following cases:

• Brachial plexus injury

• Vasculitis of the extremities.

Precautions

• Strictly comply with your healthcare professional's prescription and recommendations for use.

• Take care not to damage the device when fitting it, particularly with your nails.

• LYPHATREX Expert compression devices are designed to be worn during the day.

• These devices are made to the patient's measures to exert the necessary pressure recommended by the healthcare professional. They should not be used by another person as this could present risks (hygiene, safety and performance).

• Achieving the necessary pressure may require the overlaying of several compression devices.

• Treatment success requires that the device be worn regularly.

• Do not apply the product directly to an open wound.

• Before using a compression device, ensure that any skin lesions are fully covered with an appropriate dressing.

• Do not apply creams before putting on the garment since these could damage the product.

• Ensure that the anti-slip system can be applied to healthy undamaged skin.

• In the event of significant discomfort, changes in limb volume or unusual sensations, deterioration of skin condition, infection or a change in performance of the device, remove the device and seek the advice of a healthcare professional.

• Due to the constantly evolving nature of the condition, regular monitoring by a healthcare professional is recommended to ensure that the garment is still suitable for your condition.

Contraindications

Do not use LYPHATREX Expert in the following cases:

• Weeping or major dermatological conditions of a limb concerned

• Known allergies to the materials used

- Do not use when bathing.

Undesirable side-effect

This device can cause skin reactions, such as redness, itching, blistering, burns or blisters.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Washing

- It is recommended that the garment be washed before it is used for the first time.
- Wash your garment every day, or after each use.
- LYMPHATREX Expert compression devices are designed for daily use for periods of up to 6 months (i.e. 90 washes).
- They should then be replaced to maintain their technical characteristics and the effectiveness of the treatment.

Machine washing:

- Machine wash at 30°C using the delicates program.
- Use a washing product without softener. Do not use aggressive products (e.g., products containing chlorine).
- Place the garment in a wash bag to make sure other garments (particularly those with touch fasteners or zips) do not damage the knit of your orthosis.
- Squeeze out excess water without twisting.
- Dry well away from any direct heat source (radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

Storage

Store the device at room temperature and away from direct sunlight.

Disposal

LYMPHATREX Expert compression devices should be disposed of as per local regulations.

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©Thuasne - 2021501 (2020-11)