THUASNE

MalleoSoft

fr	Chevillère élastique proprioceptive	7
en	Proprioceptive elastic ankle support	8
de	Elastische propriozeptive Knöchelbandage	9
nl	Proprioceptieve elastische enkelbrace	11
it	Cavigliera elastica proprioceittiva	12
es	Tobillera elástica propioceptiva	14
pt	Tornoeleira elástica propioceptiva	15
da	Proprioceptisk, elastisk ankelbind	17
fi	Proprioseptiivinen joustava nilkkatuki	18
sv	Proprioceptiv elastisk ankelortos	20
el	Ελαστική ιδιοδεκτική επιστραγαλίδα	21
cs	Elastická proprioceptivní kotníková ortéza	22
pl	Elastyczna proprioceptywna orteza skokowo-stopowa	24
lv	Proprioceptīvs elastīgs potītes pārsējs	25
lt	Tamprus propriocepcija gerinantis čiurnos įtvaras	27
et	Propriotseptiivne elastne hüppeliigesetugi	28
sl	Elastična proprioceptivna opornica	30
sk	Elastická proprioceptívna členková ortéza	31
hu	Proprioceptív rugalmas bokarögzítő	33
bg	Проприоцептивна еластична шина за глезен	34
ro	Glezneră elastică proprioceptivă	35
ru	Эластичный проприоцептивный ортез на лодыжку	37
hr	Elastični proprioceptivni steznik za gležanj	38
zh	彈力本体感护踝	40
ar	دعامة كاحل مرنة متعلقة باستقبال الحس العميق	43

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	脚踝尺寸
ar	محيط الكاحل

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en

PROPRIOCEPTIVE ELASTIC ANKLE SUPPORT

Description/Destination

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - Cotton - Elastane.

Properties/Mode of action

Elastic compressive knitting (class 2 - 15-20 mmHg / 20,1-27 hPa).

Comfort zone over the ankle bones.

Indications

Post-operative, post-traumatic residual pain and/or edema.

Proprioceptive support for rehabilitation.

Sport activities resumption.

Compression for scar/burns healing.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

Care/Maintenance

Product can be washed in accordance with the conditions shown on this notice and on the label.

Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2023901 (2021-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros