**THUASNE**

# MOBIDERM autofit


<b>fr</b> Vêtements mobilisateurs ajustables pour les œdèmes.....	8
<b>en</b> Adjustable mobilizing garments for edema.....	8
<b>de</b> Verstellbar mobilisierende Kleidung zur Behandlung von Ödemen.....	9
<b>nl</b> Verstelbare mobilisatiekleding voor oedemen.....	9
<b>it</b> Indumenti di mobilizzazione regolabili per edemi.....	10
<b>es</b> Prendas movilizadoras ajustables para edemas.....	10
<b>pt</b> Roupas de mobilização ajustáveis para os edemas.....	11
<b>da</b> Mobiliserende og justerbare beklædningsgenstande til ødemer.....	11
<b>fi</b> Säädettävät mobilisoivat vaatteet turvotukseen.....	12
<b>sv</b> Justerbara mobiliserande plagg för ödem.....	13
<b>el</b> Ρυθμιζόμενα ενδύματα κινητοποίησης για τα οιδήματα.....	13
<b>cs</b> Nastavitelné mobilizační prádlo pro léčbu edémů.....	14
<b>pl</b> Regulowana odzież mobilizująca na obrzęki.....	14
<b>lv</b> Regulējams mobilizējošs apģērbs tūskām.....	15
<b>lt</b> Poodinę cirkuliaciją skatinantys reguliuojami drabužiai ištinusioms vietoms.....	15
<b>et</b> Nahaaluseid kudesid mobiliseerivad kohandatavad riideesemed ödeemide puhuks.....	16
<b>sl</b> Prilagodljiva mobilizacijska oblačila za edeme.....	16
<b>sk</b> Nastaviteľné mobilizačné návleky na liečbu lymfedému.....	17
<b>hu</b> Állítható mobilizáló ödémacsökkentő öltözet.....	17
<b>bg</b> Регулируеми мобилизираци дрехи за отоци.....	18
<b>ro</b> Îmbrăcăminte de mobilizare ajustabilă pentru edeme.....	18
<b>ru</b> Мобилизующая одежда регулируемого размера для лечения отеков.....	19
<b>hr</b> Prilagodljivi odjevni predmeti za imobilizaciju kod edema.....	20
<b>zh</b> 用于水肿治疗的活动治疗衣.....	20
<b>ja</b> リンパ浮腫用調整可能活性化衣服.....	21
<b>ko</b> 조절 가능한 부종 모빌라이징 가먼트.....	21
<b>ar</b> أغطية مُحركة للوذمات قابلة للتعديل.....	22

fr	Chaussette	nl	Kous	pt	Meia até ao joelho	sv	Knästrumpa	pl	Skarpetka	et	Sokk	hu	Zokni	ru	Носок	ja	靴下
en	Below-knee	it	Calza corta	da	Strømpe	el	Κάλτσα	lv	Īsā zeķe	sl	Nogavica	bg	Чопан	hr	Čarapa	ko	양말
de	Wadenstrumpf	es	Calcetin	fi	Sukka	cs	Podkolenka	lt	Kojinė	sk	Lýtkový návlek	ro	Șosetă	zh	袜子	ar	جوارب

	cB (cm)	cC (cm)	cD (cm)	ℓD (cm)			
1	16 - 24	24 - 35	25 - 36	S 38 - 40	N 40 - 42	L 42 - 44	XL 44 - 46
2	20 - 28	32 - 43	32 - 43				
3	24 - 32	40 - 51	39 - 50				
4	28 - 36	48 - 59	46 - 57				

fr	Bas-cuisse	nl	Dijkous	pt	Meia até à coxa	sv	Lårstrumpa	pl	Pończocha	et	Reieosa	hu	Combfix	ru	Чуллок	ja	ロングストッキング
en	Thigh-high	it	Calza alla coscia	da	Lårstrømpe	el	Κάλτσα ριζομηρίου	lv	Garā zeķe	sl	Nadkolenka	bg	Чопан за бедрото	hr	Čarapa do visine bedra	ko	허벅지 스타킹
de	Oberschenkelstrumpf	es	Media-muslo	fi	Reisisukka	cs	Stehenní punčocha	lt	Kojinė iki klubo	sk	Stehenný návlek	ro	Ciorapi	zh	长筒袜	ar	أسفل الفخذ

	cB (cm)	cC (cm)	cG (cm)	ℓG (cm)			
1	16 - 24	24 - 35	46 - 57	S 67 - 72	N 72 - 77	L 77 - 82	XL 82 - 87
2	20 - 28	32 - 43	53 - 65				
3	24 - 32	40 - 51	61 - 72				
4	28 - 36	48 - 59	68 - 80				

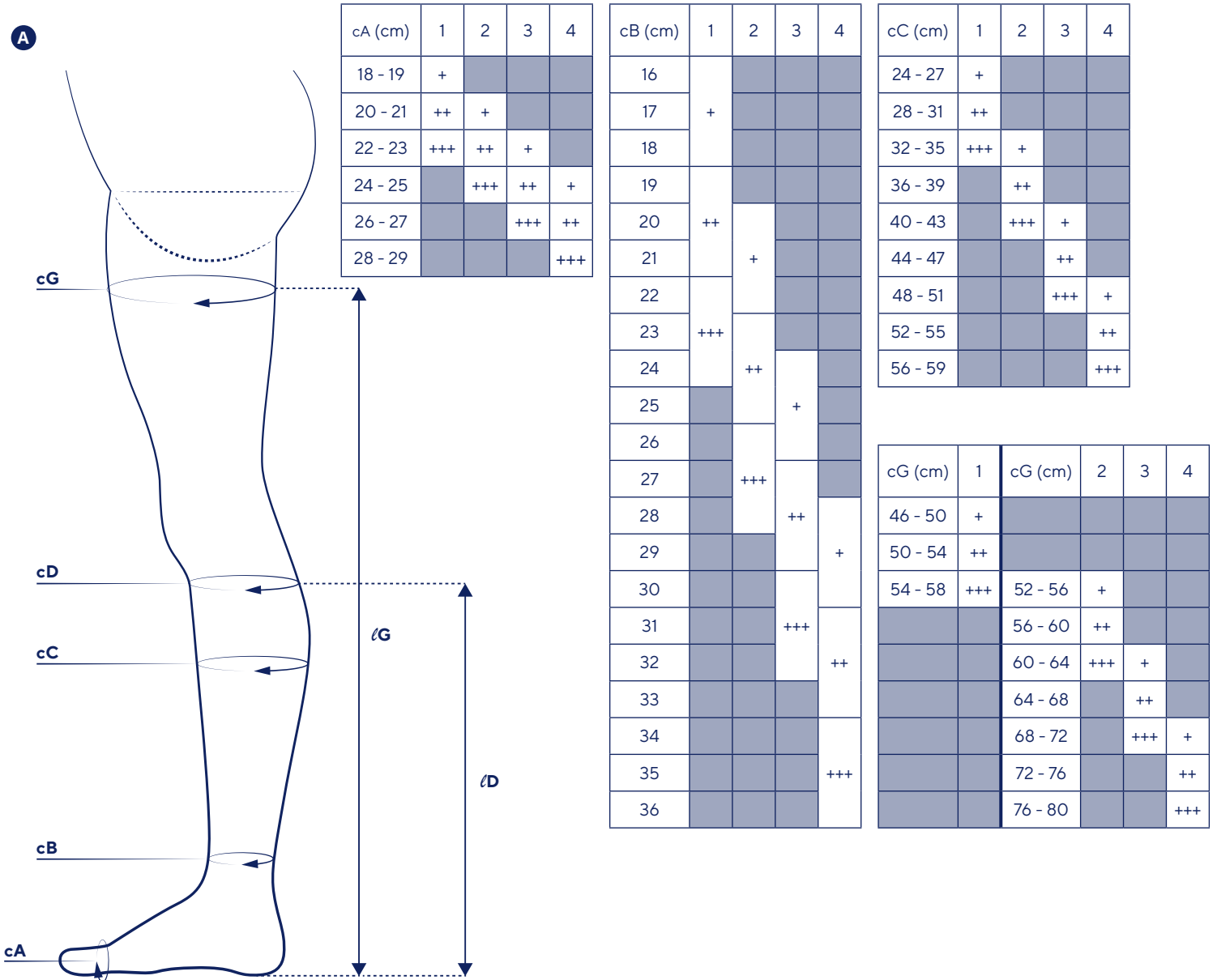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Conserver cette notice.

## en

## ADJUSTABLE MOBILIZING GARMENTS FOR EDEMA

### Description

MOBIDERM AUTOFIT garments are a range of adjustable mobilizing devices for the treatment of edema. This range is available in several models:

- Armsleeve with mitten
- Thigh-high (ambidextrous)
- Below-knee (ambidextrous)

The garments are composed:

- On the internal surface, of MOBIDERM technology consisting of foam blocks (polyurethane for the big blocks and/or polyethylene for the small blocks) encased between two sheets of non-woven material (polyester, acrylic adhesive mass);
- On the external surface, of two beige-colored, soft, elastic materials (polyester, cotton, elastane, polyamide) and hook-and-loop fasteners (PVC, polyamide, polyurethane), allowing the garment to be adjusted to limb volume.

Two sizes of foam block are used depending on the body area treated: small blocks (5 x 5 mm) on the hand and foot, big blocks (15 x 15 mm) on the leg and arm.

### Mechanism of action

MOBIDERM AUTOFIT garments provide:

- compression by applying pressure to the limb, thus promoting the circulation of lymph and helping to maintain or reduce swelling.
- subcutaneous drainage via MOBIDERM technology consisting of foam blocks positioned randomly between two sheets of non-woven material. In contact with the skin, the blocks provide a pressure differential between the contact area beneath them and their periphery, creating multi-directional circulation channels for lymph under the skin. The lymph travels from the areas pressed by the blocks towards the free areas. Efficient lymph drainage and circulation is thus promoted. **C**

The garments are designed to be worn day and night.

Use: MOBIDERM AUTOFIT mobilizing garments are intended for treatment of lymphedema of the upper or lower limbs.

### Indications

MOBIDERM AUTOFIT mobilizing garments are indicated for the maintenance of the reduction in lymphedema swelling.

Target population: adult patients and children suffering from indicated pathologies and compatible with the size table.

### Fitting **D**

-  It is recommended that your garment be washed before it is used for the first time.
- Always make sure that the product is intact before each use.
- The MOBIDERM AUTOFIT garment can be put on next to the skin like a garment or over a cotton bandage for fragile skin.

#### Donning of below-knee and thigh-high:

- Open all the hook-and-loop fasteners on one side of the garment **1**.
- Insert the foot into the garment **2** then slide the garment carefully along the limb to below the knee (for below-knee model) **3** or to below the gluteal fold (for thigh-high model).
- Avoid pulling on the garment more than is necessary.
- Close the hook-and-loop fasteners beginning at the foot **4** and working up to the knee or the gluteal fold **5**.
- Proceed in several stages until the adjustment recommended by your health care professional is achieved (+/+/+/+/+/+/+/+) **A**.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **6**.

#### Donning of armsleeve with mitten:

- Open all the hook-and-loop fasteners **7**.
- Insert the hand into the garment **8** then slide the garment carefully along the limb up to the armpit **9**.
- Avoid pulling on the garment more than is necessary.
- Close the hook-and-loop fasteners beginning at the hand **10** and working up to below the armpit **11**.
- Proceed in several stages until the adjustment recommended by your health care professional is achieved (+/+/+/+/+/+/+/+) **B**.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **12**.

#### Adjustment of hook-and-loop fasteners **A B**

MOBIDERM AUTOFIT garments display tightening indications (+/+/+/+/+/+/+/+) at several points. These allow the garment to be adjusted to your morphology depending on the size of the garment that has been issued to you. Recommendations for adjustment of the hook-and-loop fasteners are also given at the start of the instruction leaflet for informational purposes. In all cases refer to the recommendations of the health care professional who prescribed the product.

### Contraindications

General:

- Skin infections of the limb or acute inflammation.
- Known allergies to any of the components.
- Septic thrombosis.
- Severe peripheral neuropathy of the limb.

Specific to the lower limbs:

- Lower limb peripheral arterial disease (PAD) with ankle brachial index (ABI) < 0.6.
- Decompensated heart failure.
- Advanced diabetic microangiopathy, phlegmasia cerulea dolens (painful blue oedema with arterial compression).
- Extra-anatomic bypass

Specific to the upper limbs:

- Brachial plexus pathology.
- Vasculitis of the extremities.

### Precautions

- Follow the advice of a health care professional regarding treatment duration and methods.
- During the lymphedema maintenance phase, the MOBIDERM AUTOFIT garment should be worn at night in alternation with the compression garment worn during the day. It should be worn throughout the night. It can be worn during the day during rest phases.
- Do not use the product directly on damaged skin.
- Application of this device can cause the appearance of marks on the skin. These marks are benign and temporary; do not stop treatment without medical advice.
- Fully cover any skin lesions with an appropriate dressing before putting on the garment.
- If you experience significant hindrance, discomfort, pain, skin damage, infection, unusual sensations (example: tingling), coloring of the extremities, or change in performance, remove the device and consult a health care professional.
- For fragile skin, we recommend applying a cotton bandage to the limb before putting on the garment.
- If the edema worsens, consult a health care professional.
- Due to the constantly changing nature of this condition, we recommend regular check-ups with a health care professional to ensure the device is still suitable for you.
- Do not use when swimming. Do not expose to a heat source or seawater.
- Do not apply creams before putting on the garment as they may damage the product.
- Take care not to damage the device with your nails when fitting it.

- If you suffer from advanced venous disease or ulcers, consult a health care professional for advice on a suitable full-leg compression garment.
- For reasons of hygiene, safety and performance, do not share your garment with anyone else. The product should not be reused for another patient.

#### Undesirable side-effect

- Skin reactions: blisters, itching, marks, burns and redness.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### Care/Maintenance

To ensure your MOBIDERM AUTOFIT garments last as long as possible and to prevent infection, it is important to wash them every day, or after each use.

Close the hook-and-loop fasteners before washing.

Hand wash: soak the device in cold soapy water, rub with a clean sponge and squeeze out excess water. Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

In a washing machine:

- Use the wool cycle at a cool temperature.
- Use a laundry detergent without fabric softener.
- Do not use chlorine-based detergents or harsh products.
- Place the garment in a mesh laundry bag to prevent damage by other garments (particularly with self-fastening tabs or zippers).
- Squeeze out by hand.
- Air dry, away from direct heat (e.g. radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

When the garment is not being worn, preferably keep it flat, with the hook-and-loop fasteners closed . If it has to be folded, position the hooks on the outside .

#### Disposal

Check local regulations for how to dispose of MOBIDERM AUTOFIT garments.

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©Thuasne - 2017801 (2020-10)