THUASNE

Pod-med

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Pressure-relieving foot orthoses

Purpose

Podo-med foot support orthosis is a made-to-measure therapy shoe for relieving strain at either the front or back of the foot. The shoes may only be used for foot treatment.

Indications

- **50030 Forefoot short version**
- **50031 Forefoot short version with toe guard:**

Postoperatively, after forefoot surgery, patients with plantar ulcers, trauma and fractures.

- **50040 Forefoot long version with protective toe ridge**
- **50045 Forefoot long version with insole**
- **50047 Forefoot long version with additional guide strap:**

Postoperatively, patients with plantar ulcers, trauma and fractures.

- **50050 Open heel version**
- **50051 Open heel version with heel cover:**

Postoperatively, patients with plantar ulcers, trauma and fractures.

Contraindications

Not suitable for patients whose foot soft tissue does not tolerate extra load.

Side Effects

None reported when applied and worn as intended.

Information for patients

Your doctor or orthotist has prescribed a valuable item of orthopaedic footwear for you.

In order to benefit from it, and for your own comfort and safety, please observe the following points before and during use

- All open wounds on the foot must be protected by a bandage.
- We recommend that you are seated whilst putting on and taking off your shoe.
- Always wear a slip resistant shoe on your healthy foot, providing you with equal heel height.
- Do not attempt to ride a bike, drive a car or other vehicle whilst wearing Podo-med.

If you are a diabetic patient you should observe the following points when wearing this product

- Take care to inspect the shoe for possible pressure points several times a day in order to avoid any skin lesions.

- If you do become aware of pressure points please see your orthotist as soon as possible.
- Should you develop any skin lesions due to pressure points please visit your doctor without delay.

Information for the orthotist/orthopaedic technician

The protective toe guard, part of the short forefoot-pressure-relief version, can be reshaped at 140°C. Please use heat resistant gloves for this procedure.

Application Instructions

- (1) Open all touch-and-close fasteners and remove all straps.
- (2) Insert your foot into the open Podo-med shoe.
- (3) Product Code 50047 requires you to close the front strap first. (N.B. This product can be worn with or without the additional insole).
- (4) Where a U-shaped fastening strap is supplied (with all forefoot versions) always position it in such a way that the open ends point to the left if the shoe is worn on the left foot, or to the right hand side if worn on the right.
- (5) Guide and feed the single strap through the loop on the opposite side of shoe and close fastening. Proceed in the same way with remaining other strap(s). Ensure a firm but not tight fit. Any excess strap material can be cut off.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care Instructions

- These shoes can be hand washed at 30°C, using a mild detergent
- Remove all detachable straps and fasten all fixed ones

Composition

Polyamide, soft polyurethanes, PU foam, EVA foam, EVA poro rubber

Please keep this instruction leaflet

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH