THUASNE®

Rhizo-pro

- de **Gebrauchsanweisung - Daumensattelgelenk-Orthese**
- en **Instructions for use - Wrist/thumb orthosis**
- fr **Consignes d'utilisation - Orthèse statique poignet pouce**
- it **Istruzioni per l'uso - Ortesi per il pollice e il metacarpo**
- es **Instrucciones de utilización - Órtesis de la articulación trapeciometacarpiana**
- pt **Instruções de uso - Órtese para a articulação trapeziometacarpiana**
- nl **Gebruiksaanwijzing - Duim-handwortel-Orthese**
- da **Brugsanvisning - Tommelfingerortose**
- sv **Bruksanvisning - Tumortos**
- fi **Käyttöohje - Peukalon tyvinivelen tukiside**
- el **Οδηγίες χρήσης - Νάρθηκας επιπλοειδούς άρθρωσης αντίχειρα**
- cs **Návod k použití - Ortéza na palec ruky**
- pl **Sposób użytkowania - Orteza stawu siodełkowatego kciuka**
- et **Kasutusjuhend - Pöidlapõhimiku liigese ortoos**
- lv **Lietošanas instrukcija - Plaukstpamata un īkšķa locītavas ortoze**
- lt **Naudojimo instrukcija - Riešo ir nykščio įtvaras**
- sk **Návod na použitie - Ortéza na palec ruky**
- sl **Navodila za uporabo - Orteza za sedlasti sklep palca**
- hu **Használati utasítás - Hüvelykujj nyeregízületi ortézis**
- ro **Instrucțiuni de utilizare - Orteză metacarpo-falangiană pentru policeu**
- bg **Инструкция за употреба - Ортеза за седловидната става на палеца**
- hr **Upute za uporabu - Ortoza za sedlasti zglob palca**

Größe ¹	 Handgelenkumfang ²		 Daumenbreite ³	
	cm	inch	cm	inch
1	12,0 - 18,5	4 ¾ - 7 ¼	1,7 - 2,2	⅔ - ⅘
2	16,5 - 22,0	6 ½ - 8 ⅔	2,2 - 2,6	⅘ - 1 ⅕

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45 % Polyamid, 32 % Elasthan, 23 % Polyester

Wrist/thumb orthosis**Purpose:**

Rhizo-pro is a thumb orthosis used for immobilisation. The orthosis may only be used for thumb treatment.

Mode of action:

Reliable immobilisation of the carpal and first carpo-metacarpal (CMC I) joints as well as the metacarpo-phalangeal (MCP I) joint of the thumb. The anatomically correct shape of the product ensures a comfortable, secure fit.

Indications:

Arthrosis/arthritis, distortions, injuries (trauma), instabilities of the ligaments, postoperative

Contraindications:

Skin trauma or disease in the thumb or wrist area

Side Effects:

No side effects are as yet known for correct use.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not reuse the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Fitting instructions:

1. Open the touch-and-close fastener on the support.
 2. Place the thumb fully into the orthosis.
 3. Wrap the wrist strap around your wrist joint and secure.
 4. Wrap the thumb strap around your thumb and secure.
- The orthosis should lend firm support without cutting in.

Care Instructions:

To clean the orthosis, remove the plastic shell from the textile part. Close Velcro straps before washing. Clean the plastic shell with a damp cloth and a little washing-up liquid.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
UK Responsible Person (UKRP): Thuasne UK Ltd
Thuasne Deutschland GmbH