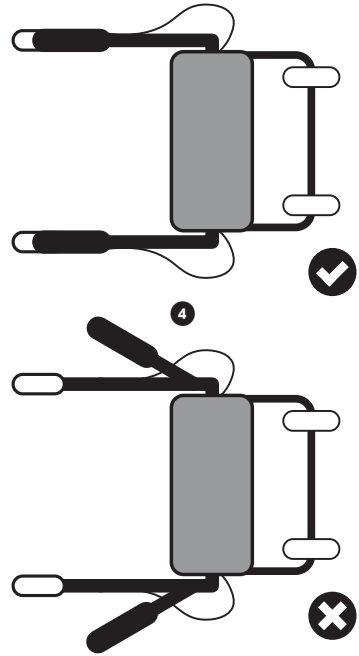
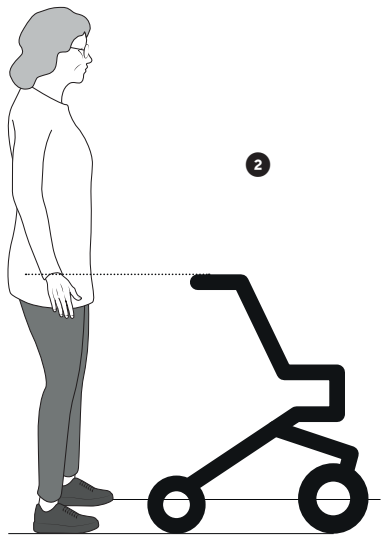
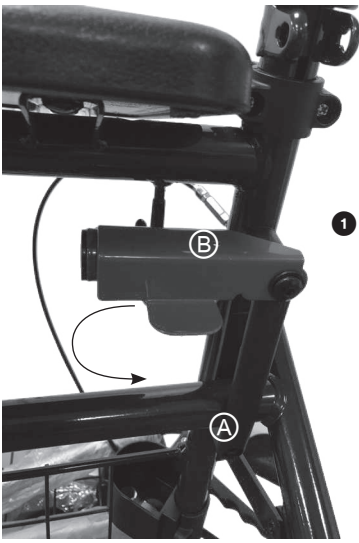
**THUASNE**

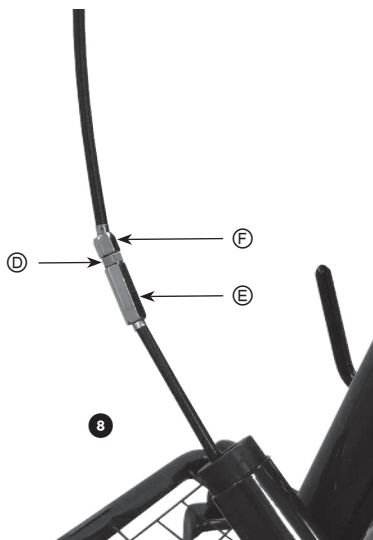
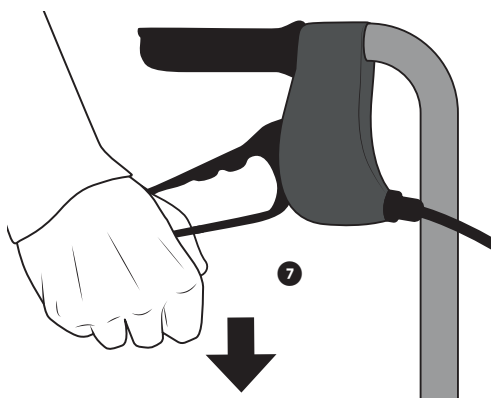
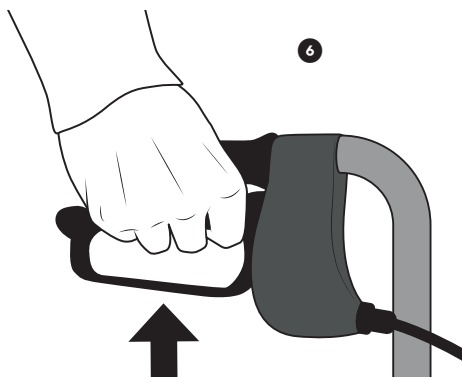
# Quatro

V05040

fr	Rollateur .....	4
en	Rollator .....	6
de	Rollator .....	8
nl	Rollator .....	10
it	Rollator .....	12
es	Andador .....	14
cs	Chodítko .....	16
pl	Podpórka .....	18
sk	Štvorkolka .....	20

**MD**

<b>fr</b>	Dispositif médical
<b>en</b>	Medical device
<b>de</b>	Medizinprodukt
<b>nl</b>	Medisch hulpmiddel
<b>it</b>	Dispositivo medico
<b>es</b>	Producto sanitario
<b>cs</b>	Zdravotnický prostředek
<b>pl</b>	Wyrób medyczny
<b>sk</b>	Zdravotnícka pomôcka

<b>fr</b>	Lire attentivement la notice
<b>en</b>	Read the instruction leaflet carefully
<b>de</b>	Lesen Sie die Gebrauchsanweisung sorgfältig durch
<b>nl</b>	Lees voor gebruik de gebruiksaanwijzing aandachtig door
<b>it</b>	Leggere attentamente le istruzioni
<b>es</b>	Leer atentamente las instrucciones
<b>cs</b>	Pozorně si přečtete návod
<b>pl</b>	Należy dokładnie przeczytać instrukcję
<b>sk</b>	Tento návod si pozorne prečítajte

# ROLLATEUR QUATRO

## Description/Destination

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## Description/Destination

4-wheel rollator.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It also has a seat for taking short rests. It can be used inside and outside.

## Main components

Frame: steel

Handles: PVC

Seat (cover): polyurethane

Wheels/tires: polypropylene + thermoplastic rubber

Tray: plastic

Basket: plastified steel

## Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.

Degenerative conditions of the lower limbs.

## Contraindications

Not suitable for patients weighing > 130 kg.

Do not use in the event of known allergy to any of the components.

Do not use in the event of severe perceptual function deficit or severe balance disorders.

## Precautions

### Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Before using this product, check that:

- it is stable and rigid,
- the brakes are working properly,
- the wheels are in good condition and turn freely,
- no components are incorrectly attached or not properly screwed in place,
- the locking levers on the handles are properly tightened and pointing downwards.

The health care professional who supplied the product must make sure that it is appropriate for the patient.

It is recommended that a health care professional assembles and adjusts the device and supervises the first use. In particular, it is essential that they check and adjust the tension of the brake cables.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Do not use the device for any use other than its intended use: risk of injury or damage to equipment.

Never allow children to play with the product.

Beware of the risks of pinching when handling the device.

Always completely unfold the rollator before use.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven, sloping or unstable surfaces (gravel, sand, leaves, etc.).

Do not use on stairs or escalators.

Do not leave exposed to direct sunlight for a long period of time. Check the temperature of the device surfaces, particularly before sitting down on the seat: risk of burns.

## Undesirable side-effect

Risk of discomfort in the event of incorrect adjustment of the device.

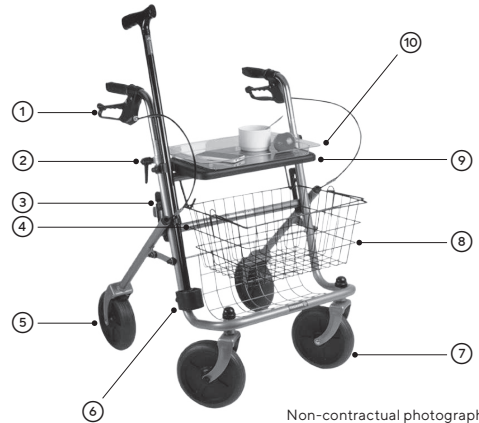
Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## Instructions for use

The rollator is supplied with a basket, a small luggage rack, a tray and a cane holder.

- ① Brake
- ② Tightening lever for adjustment of the handle height
- ③ Anti-folding lock (red part)
- ④ Cross tube
- ⑤ Rear wheel with brake
- ⑥ Cane holder
- ⑦ Pivoting front wheel
- ⑧ Basket
- ⑨ Seat
- ⑩ Tray

Non-contractual photograph.  
Cane not supplied.

## Assembly/unfolding

### 1. Opening the rollator

Place the rollator on its wheels, pull the back of the frame and firmly push the cross tube downwards to completely open the rollator.

The cross tube should slide to the bottom of the plastic bracket (A). Then pivot the red lever (B) downwards to lock the tube ①.

### 2. Assembling the handles/adjusting the height

The height of the device must be adjusted with reference to the height of the patient. Incorrect adjustment can impair performance and place too much strain on the user. The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes ②.

• Insert the handle rods into the tubes on either side of the seat for a length of at least 10 cm. Insert the tightening screws, without tightening, making sure to push them in fully.

• **Maximum** height adjustment: the handle rods should not be positioned beyond the mark located next to the "SECURITY" indication ③.

• The handles should be parallel, they should not be turned outwards ④.

- Tighten the tightening screws firmly by turning clockwise. The locking levers should be pointing downwards: if necessary, pull them outwards and pivot downwards **5**. Make sure that the handles are fixed at the same height on both sides. Then press the handles firmly down towards the ground and check that all 4 wheels are in contact with the ground.

### 3. Installing the accessories

Tray: place it on the seat, taking care to position the pins in the holes.

Basket: suspend it from the two hooks located under the seat at the front of the rollator.

### Folding

1. Remove the tray and basket.
2. Release the brakes, deactivate the anti-folding lock by pivoting the red lever upwards and raise the cross tube to fold the rollator.
3. Then lock the brakes so that the rollator cannot move. Once folded, the rollator stands on its own. To carry the rollator safely, hold it by the cross tube

### Braking system

- Quick braking  
To brake, hold on to the handles and squeeze the brake levers. **6**
- Parking brakes  
To immobilize the rollator (e.g., before sitting down), carefully push the brake levers down until they lock (click). **7**  
To start moving again, gently squeeze the levers to release the brakes.
- Adjusting the brakes  
- If braking is not sufficient, loosen the locknut **D** holding the support **E** then turn the adjustment screw **F** anticlockwise **8**.  
- If braking is too hard, proceed as above but turn the screw clockwise **8**.  
Remember to tighten the locknut again after adjusting to the optimal setting.  
If necessary, adjust the setting at the screws on the front of the handles.

### Important warnings

Before using the seat:

- check that the rollator is on a flat, stable surface,
- it is essential for the brakes to be in the "parking brakes" position.

Do not sit at an angle or lean to the side or back.

The rollator seat is only intended for taking short rests.

Do not attach handbags, carrier bags or any objects other than those supplied with the device: risk of tipping.

The device must not be used as an aid to get up from the sitting position: risk of unbalancing.

### Maintenance and care

#### Maintenance

If any part of the device becomes detached, loose or visibly damaged, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from your Thusne product dealer.

#### Care

Clean with a mild cleaning agent and a soft cloth. Do not clean the handles or the wheels with greasy substances: risk of slipping. If necessary, use a mild contact disinfectant or 70° alcohol. In an institution, follow the recommendations of the hygiene manager.

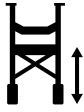
### Storage

Store at room temperature, in a dry, ventilated place.

### Disposal

Dispose of in accordance with local regulations.

### Technical characteristics

Handle adjustment height  78.5 - 97.5 cm	Total width  59.5 cm	Total depth  67.5 cm
Seat height  60.5 cm	Seat width  39 cm	Seat depth  22 cm
Basket load  5 kg max	Tray load  5 kg max	Luggage rack load  10 kg max
<b>Maximum total load: 10 kg max</b>		
Total weight without/with basket and tray  9.4 kg/10.2 kg	Maximum user weight  130 kg max	

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©Thuasne - 2026101 (2021-02)