


MOBIDERM ARMSLEEVE


POSITION OF THE ANATOMICAL MARKERS

- G - corresponds to the axilla (armpit)
- F - corresponds to the widest point of the biceps
- E - is located at the level of the olecranon
- D - equidistant between A and C or at the starting site of the oedema
- C - to be measured at the narrowest point of the wrist external face in the middle of the radial ulnar joint

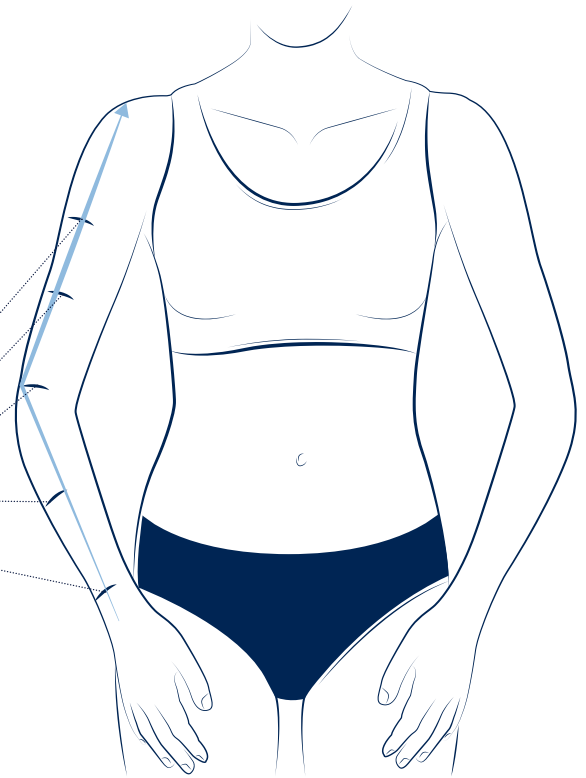


→ Positioning of the measuring tape to measure heights

ARMSLEEVE

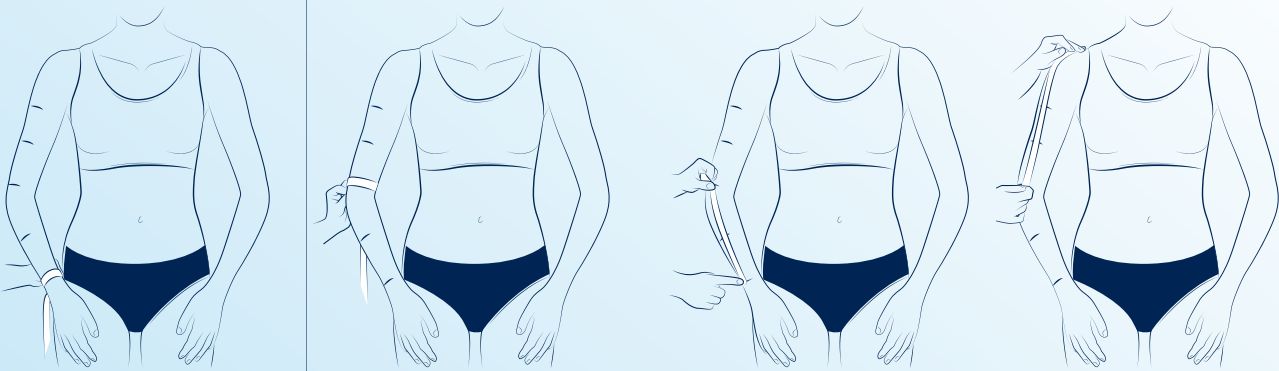

The lengths must be measured from the middle of the wrist to the tip of the elbow, then at the outside of the arm up to the desired finished height.

MITTEN

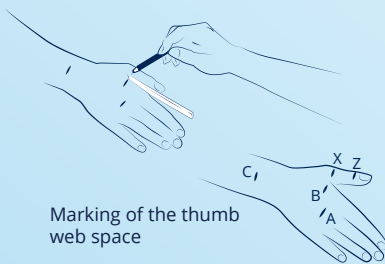

Marking of the thumb web space



Recording of marker points (thumb web space): measure the circumference at the base of the thumb and at the end. For technical reasons, the length of the thumb cannot be less than 2 cm.



The circumference at the end of the mitten is located at the head of the metacarpal bones. Switch to Cicatrex if the fingers also need pressure



Measuring the length of the mitten:
 1 / from the wrist to the base of the thumb
 2 / from the wrist to the finished height